

FOLDING ALUMINIUM CANE

6814, 7912, 7913, 7914, 7915, 7916

Instructions

- Always adjust the walking stick to the correct height before use. There are 5 different height settings.
- The 4 sections of the walking stick will snap into place once the elastic band is removed.
- To dismantle, pull out each section individually, fold and secure.

WARNING:

- Always check these sections are securely in place before use.
- Do not use under the influence of drugs or alcohol.
- Maximum weight load limit: **100kg**
- Please contact your medical practitioner before use, if you have any doubts to the suitability of the walking stick to your specific, personal needs and how best for you to use the item in regard to your own needs.

PACKAGING:

- Dispose of the packaging at your local recycling centre.
- Dispose of paper and carton separately from plastic bags at your local recycling centre.
- Dispose of the item at the end of its lifespan at your local authorised household waste recycling centre.

Head Office: Customer Services:
11 Bridge Street Tel: 0844 482 4400
Bishops Stortford Fax: 01279 756595
Hertfordshire
CM23 2JU

Coopers
OF STORTFORD