



Toe Tubes (2) Instruction Sheet

Our Toe Tubes are safely attached to the toes and filled with a layer of gel. They offer excellent protection against painful corns, pustules and callous on or between the toes.

This product also protects against shocks and suits to your toes.

The tube can be cut to the suitable length and is sufficient for a total of 6 to 7 individual protection pads.

Cleaning:

For cleaning the lining, turn the pads inside out and wash them in mild soapy water, rub them gently and let them air dry. Do not bleach.

Caution: The pads should not be worn longer than 20 hours a day.

Diabetics or people with bad blood circulation should contact a doctor before use.

Cover Material: 60% Polyamide, 40% Elastane Inner: 100% TPR

