Imported by - Importé par:  
 GD import (Genius Ideas & Welllys)  
 190 chemin de la Frayère - 06530 Peymeinade - France  
[www.gdimport.com](http://www.gdimport.com)  
[www.genius-ideas.com](http://www.genius-ideas.com)  
[www.wellys-care.com](http://www.wellys-care.com)

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#### GB - SUPER EAR "BLUELOOK 3IN1"

Please carefully read through the operating instructions prior to using the device for the first time and store these instructions in a safe place. The instructions are to accompany the device if it is passed on to others. This device is not designed to be used by persons (including children) with impaired physical, sensory or mental abilities or those lacking experience and/or knowledge unless they are supervised by a person responsible for their safety or received instructions on how to use the device from them.

The hearing amplifier is a great device to optimize your hearing. More or less miniaturized, it filters the sound to make it more crystalline and profound. With its adjustable volume it offers superior comfort to follow a conversation, listen to a television program with a minimized volume and hear the sounds of nature.

#### Instructions:

1. Place the desired silicone cap on the hearing amplifier or on the earphone according to the mode of intended use (see diagram A).
2. Turn the volume to minimum.
3. Insert the tip of the headset into your ear, ensuring that the device is properly positioned (see diagram B).

4. Adjust the ON/OFF switch according to your individual needs:

Switch to "LOW" for amplification of lower audio frequencies

Switch to "HI" for amplification of higher audio frequencies

NOTE: To experience the difference of these two modes, please note that you should not be within close proximity of the sounds you wish to amplify.

5. Gradually increase the setting until having received a satisfying result.

Remove the device in case of larsen (treble sound) and start again at step 2.

#### Rechargeable device with USB charger:

This super headset type G is rechargeable with the USB adapter.

1. Connect the USB adapter (Figure F) to your hearing amplifier and to a computer or standard USB converter.
2. Use only the supplied USB cable, otherwise you may damage the product.
3. To charge the headset, set the unit to "OFF".

4. When the headset is charging, the LED turns red. Full loading time may vary from 12 to 14 hours.

**Maintenance:**

USB cable and main unit: Clean with a soft damp cloth. Make sure it is completely dry before reconnecting the mains charger (electric shock). Silicone tips: Clean with warm soapy water, then rinse and thorough drying.

**Caution:**

1. This product is not waterproof. Consequently, the user has to pay attention not to let any water penetrate in it (including the rain). If some water penetrates into the sound zoomer, remove the button battery and place the product in a warm and dry place. The consequences of a water exposure are not included in the guarantee.
2. It is important to increase the volume very gradually to avoid a rough increase of the flow.
3. This product must be used in a punctual way (maximum 2 hours) and with a reasonable volume.
4. This product is intended for an indoor or outdoor use with a quiet environment (40db to 60db). It is forbidden to use the device near a noisy source exceeding 70db (ex: Hoover, television with high volume, very crowded street).
5. The normal function of the product may be disturbed by Strong Electro Magnetic Interference if under EMC testing condition of: 80-2700MHz, with 3V/m at 1kHz modulation. If so, simply reset the product to resume normal operation by following the instruction manual. In case the function could not be resumed, please keep away the interference area.

THIS PRODUCT DOESN'T REPAIR ANY EAR PROBLEM AND IS NOT SUITED FOR A PERSON SUFFERING OF A MEDIUM OR HIGH AUDITION PROBLEM. IF THIS IS YOUR CASE, YOU SHOULD RETURN THIS ARTICLE AND CONSULT A HEARING SPECIALIST WHO WILL PROVIDE YOU AN APPROPRIATE AND CUSTOM-MADE SOLUTION. THIS PRODUCT IS ONLY INTENDED TO IMPROVE THE TONE OF A DISCUSSION OR A TV PROGRAM BY BRINGING A DIFFERENT LISTENING WITH AN APPRECIABLY AMPLIFIED VOLUME (MAX. +20 / 25DB). LIKE A MUSIC PLAYER: A FULL POWER USE, A NOISY ENVIRONMENT (MORE THAN 70db), OR A TOO LONG LISTENING EXPOSURE CAN DAMAGE THE USER'S EAR. USE WITH A MODERATED VOLUME FOR MAXIMUM 2 HOURS.

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GB - This product is labelled in accordance with European Directive 2012/19/EU concerning used electrical and electronic appliances (WEEE). Do not dispose of electrical appliances as unsorted municipal waste, use separate collection facilities. Contact your local government for information regarding the collection systems available. If electrical appliances are disposed of in landfills or dumps, hazardous substances can leak into the groundwater and get into the food chain, damaging your health and well-being.

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GB - Batteries and storage batteries must not be disposed of in normal household waste. As the end user you are legally responsible for returning or properly disposing of the batteries. After use you can return the batteries to the place where you bought them free of charge (e. g. at the collection point in the shop or at local authority collection points).