
Please read and retain these instructions for future reference

v002: 11/05/20

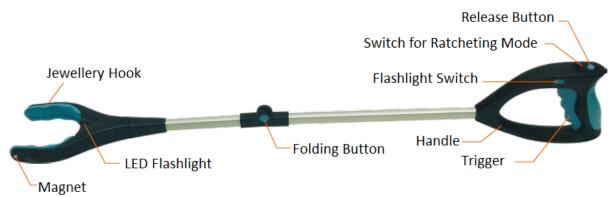


Lifts up to 2.27kg/5lbs securely and safely

How to open and close the Long Reach Grabber LED

To open and close, press and hold the blue folding button in the middle of the unit while folding or unfolding it.

IMPORTANT: Always push and hold the folding button before locking.

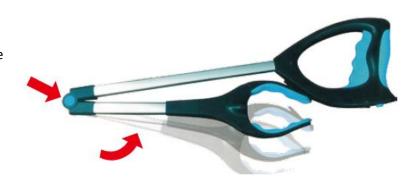

How to use the Long Reach Grabber



Make sure that the switch on the top of the handle is pushed forward. Press the blue button on the top of the handle then push the switch forward.



To use the tool as a grabber, simply place the grabbers around the object you want to pick up and squeeze and hold the trigger.



When you want to put the object down, place the object on a solid surface. Once this is in place, gently release your grip on the trigger.





Instructions for Use

Please read and retain these instructions for future reference

v002: 11/05/20

How to use the Ratcheting Mode



Make sure that the switch on the top of the handle is pushed back.



To use the ratcheting feature, simply place the grabbers around the object you want to pick up and squeeze the trigger of the handle until it is snug around the object and no longer makes a clicking sound. DO NOT OVERTIGHTEN.



Pick up and place the object on a solid surface.



Push the blue button on the top of the handle to release the object.



If you want to switch to Momentary Mode, slide the switch on the top of the handle forward until it clicks into place.

Rotating the Grabbers

To reach into tight areas, pull the grabber end out, rotate 90 degrees, and push it back in.

How to use the Flashlight

To use the flashlight, simply push the switch located on the side of the handle forwards until the light illuminates. Push it backwards to turn the flashlight off.







Please read and retain these instructions for future reference

Instructions for Use

v002: 11/05/20

Other features



The Grabber also comes with a hook, which can be used to pick up jewellery.



The Grabber has a built-in magnet, which can be used to pick up car keys, paper clips, or other small metallic objects.

Battery Replacement

The Grabber tool comes with batteries installed. First use: Remove the plastic battery tab, and you will be able to use the flashlight immediately. To replace: Remove the battery compartment cover using a Phillips screwdriver (not included). Insert 3x AG13/LR44 batteries in the polarity direction shown. Replace the cover.



BATTERIES

- Do not mix used and new batteries.
- Install batteries in accordance with its polarity symbols.
- Remove the batteries if left for long periods.
- Never dispose of batteries in a fire.
- Keep batteries away from children and pets.
- Battery disposal, spent or expired batteries must be properly disposed of and recycled in compliance with local regulations. For detailed information, contact your local authority.
- Follow the battery manufacturer's safety, usage, and disposal instructions.

Please read and retain these instructions for future reference

v002: 11/05/20

DISPOSAL

- Coopers of Stortford use recyclable or recycled packaging where possible.
- Please dispose of all packaging, paper, cartons, packaging in accordance with your local recycling regulations.



Plastics, polybags – This product contains the following recyclable plastic.

Code & Symbol	OTHER
Type of Plastic	ABS
Commonly used for	CD's, crisp packets, various flexible packaging, baby bottles, sunglass lenses
Notes	Not often or widely recycled

• At the end of the product's lifespan please check with your local council authorised household waste recycling centre for disposal.

