

**Dimensions:****Unfolded:**

80-93cm (H) x64cm (W) x68cm(D)

**Folded:**

47cm (H) x22cm (W) x65cm (D)

**Weight:** 7.9kg

**Weight Capacity:** 136kgs/300lbs

**Shopping bag dimensions:**

17cm (D) x35cm (W) x26cm (H).

**Materials:**

Aluminium, plastic (nylon and fibre)

**General Safety Warning**

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury to the user. Retain in a safe place for future reference.

**Intended Use**

The Freedom Lite Rollator is designed to provide support and assistance to those unable to, or who lack the confidence, to walk without some level of support. This rollator is intended for indoor and outdoor use, and folds away for easy storage or transportation

**Safety Precautions - Dos and Don'ts**

- DO NOT EXCEED THE MAXIMUM USER WEIGHT LIMIT OF 158KG (350LB).
- DO NOT USE THE ROLLATOR ON STAIRS OR AN ESCALATOR.
- DO NOT ATTEMPT TO PUSH THE ROLLATOR WHILE SOMEONE IS SITTING IN THE SEAT.
- DO NOT USE THE SEAT TO TRANSPORT PEOPLE OR OBJECTS.
- DO NOT USE THE SEAT WITHOUT THE BACKREST IN PLACE.

**WARNING**

**DO NOT USE THIS PRODUCT FOR ANY OTHER PURPOSE THAN AS A WALKING AID, IT IS NOT TO BE USED AS A WHEELCHAIR OF ANY KIND.**

**ONLY USE THE SEAT ONCE THE PARKING BRAKE HAS BEEN APPLIED. THE SEAT IS FOR OCCASIONAL RESTING ONLY DURING WALKING.**

**DO NOT USE THE ROLLATOR AS A WHEELCHAIR OR TRANSPORT DEVICE.**

**ROLLATORS ARE NOT INTENDED TO BE PROPELLED WHILE SEATED.**

**DO NOT ATTEMPT TO REACH OBJECTS IF YOU HAVE TO MOVE FORWARD IN THE SEAT. REACHING FOR THESE OBJECTS WILL CAUSE A CHANGE TO THE WEIGHT DISTRIBUTION OF THE ROLLATOR AND MAY CAUSE THE ROLLATOR TO TIP, RESULTING IN INJURY OR DAMAGE.**

- DO NOT SIT ON THE SEAT WHEN THE ROLLATOR IS ON AN INCLINE.
- ONLY USE THE ROLLATOR AS A WALKING SUPPORT.
- ONLY USE THE SPECIFIED ACCESSORIES AND SPARE PARTS FOR THIS ROLLATOR.
- LOADS SHOULD ONLY BE TRANSPORTED USING THE BASKET. DO NOT HANG HEAVY ITEMS FROM THE HANDLES AS THIS MAY AFFECT THE STABILITY OF THE ROLLATOR.

#### Safety Precautions - Pre Use

- ENSURE ROLLATOR IS LOCKED OPEN BEFORE USE.
- CHECK THE BRAKES BEFORE USE.
- MAKE SURE THAT ALL PARTS ARE SECURE AND THAT MOVING PARTS ARE IN GOOD WORKING ORDER.

#### Safety Precautions - During use

- DO NOT SELF-PROPEL THE ROLLATOR WHILE SEATED. SERIOUS INJURY TO THE USER AND/OR DAMAGE TO THE ROLLATOR FRAME OR WHEELS MAY RESULT FROM IMPROPER USE.
- DO NOT USE THE ROLLATOR TO WALK BACKWARDS.
- CAUTION MUST BE TAKEN WHEN NEGOTIATING KERBS AND OTHER OBSTACLES, OR WHEN USING ON SLOPING, UNEVEN OR SLIPPERY SURFACES. FAILURE TO DO SO MAY RESULT IN A SERIOUS RISK OF A FALL OR INJURY.
- DO NOT PERFORM ANY ADJUSTMENTS TO THE ROLLATOR WHILE IT IS IN USE.
- EXERCISE CAUTION WHEN THERE IS A HEAVY LOAD IN THE BASKET (MAX. LOAD 5KG).
- ALL WHEELS MUST BE IN CONTACT WITH THE FLOOR AT ALL TIMES.
- BRAKES MUST BE IN THE LOCKED POSITION BEFORE USING THE SEAT.
- WHEN USING THE ROLLATOR IN A STATIONARY POSITION, THE BRAKES MUST BE LOCKED.

#### OPENING/FOLDING THE ROLLATOR

**OPENING** (See Figure 1)



Figure 1

## INSTALLATION INSTRUCTION



Figure 2

After opening the package, insert the handle into frame (see Figure 2)



Figure 3

Insert the screws to set up the front wheels as well as both tubes for the rear wheels (see Figure 3).

## FOLDING

Hold up the seat handle to fold the rollator as shown in figure 4



Figure 4

## USING/LOCKING/UNLOCKING HAND BRAKES USING HAND BRAKES

1. Pull the brake up towards the hand grips.
2. Keep holding the brake handle to remain stationary or release brake to move rollator.

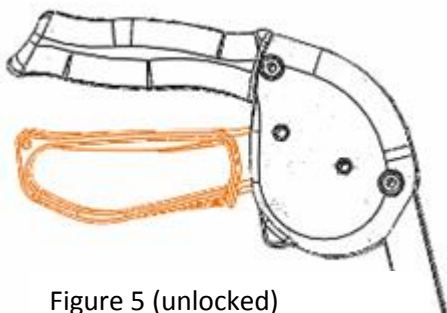


Figure 5 (unlocked)

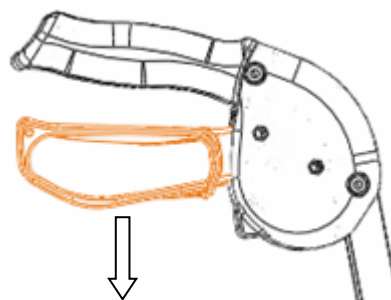


Figure 6 (locked)

1. Push DOWN on the bottom portion of the brake handle as shown until an audible “click” is heard (Figure 5 and 6)

**UNLOCKING HAND BRAKES**

1. Pull up the brake handles to release as shown in FIGURE 5.
2. Release brake handle.
3. Ensure the brake cable is fixed along the side tube and any excess cable should be rolled up in front of the rollator.

**CARE AND MAINTENANCE**

1. Ensure that ALL attaching hardware is secure at ALL times
2. Check screw to secure seat to side slot for tightness. If not, tighten if necessary.
3. Replace any broken, damaged or worn items immediately.

**CARE OF WHEELED ACCESSORIES**

1. Clean wheels with warm water and mild cleaner. Dry with a clean rag.
2. Lubricate with 3-in-1 oil periodically or if wheels begin to squeak.

**DISPOSAL**

- Dispose of all packaging, paper, cartons, plastic and plastic bags in accordance with your local recycling regulations.
- At the end of the product's lifespan please dispose of it at an authorised household waste recycling centre.