GET ME UP LADDER

Item codes: M732 / 625820

Instructions for Use

v001: 04/04/24

Please read and retain these instructions for future reference

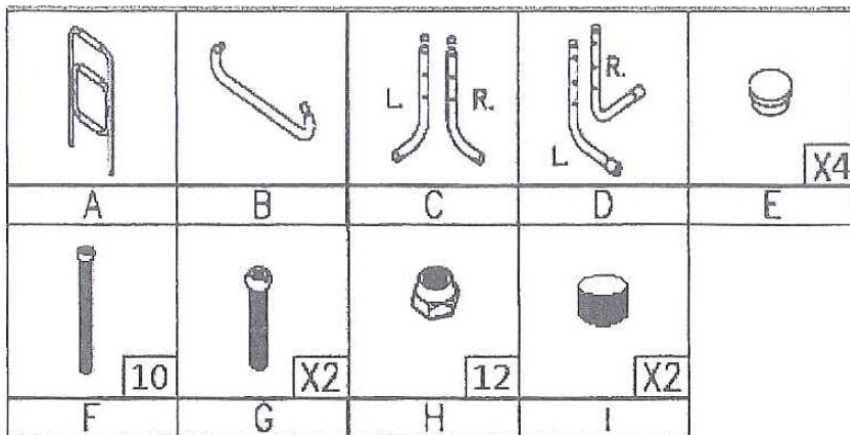
java

wellness

ENGLISH INSTRUCTIONS

INSTRUCTIONS FOR USE:

Parts:


How to assemble:

1. Slide Base (B) into Front Legs (C). See image 1.
2. Slide Screw into the opening connecting Base & Leg and the other end of the screw fasten with Nut (H). Repeat for other side. See image 1.
3. Slide Main Bar (A) onto front legs. See image 2.
4. The other end of the crew fastens and secure with Nut (H).
5. Insert Plastic Caps (E) into both ends of Main Bar (A) top ends and insert Plastic Caps (I) into bottom ends of Back Legs (D). See image 4.

Note:

- DO NOT LEAN ON OR APPLY WEIGHT.
- NOT MEANT TO BE USED AS A CANE OR WALKER.
- For support and assistance getting up.

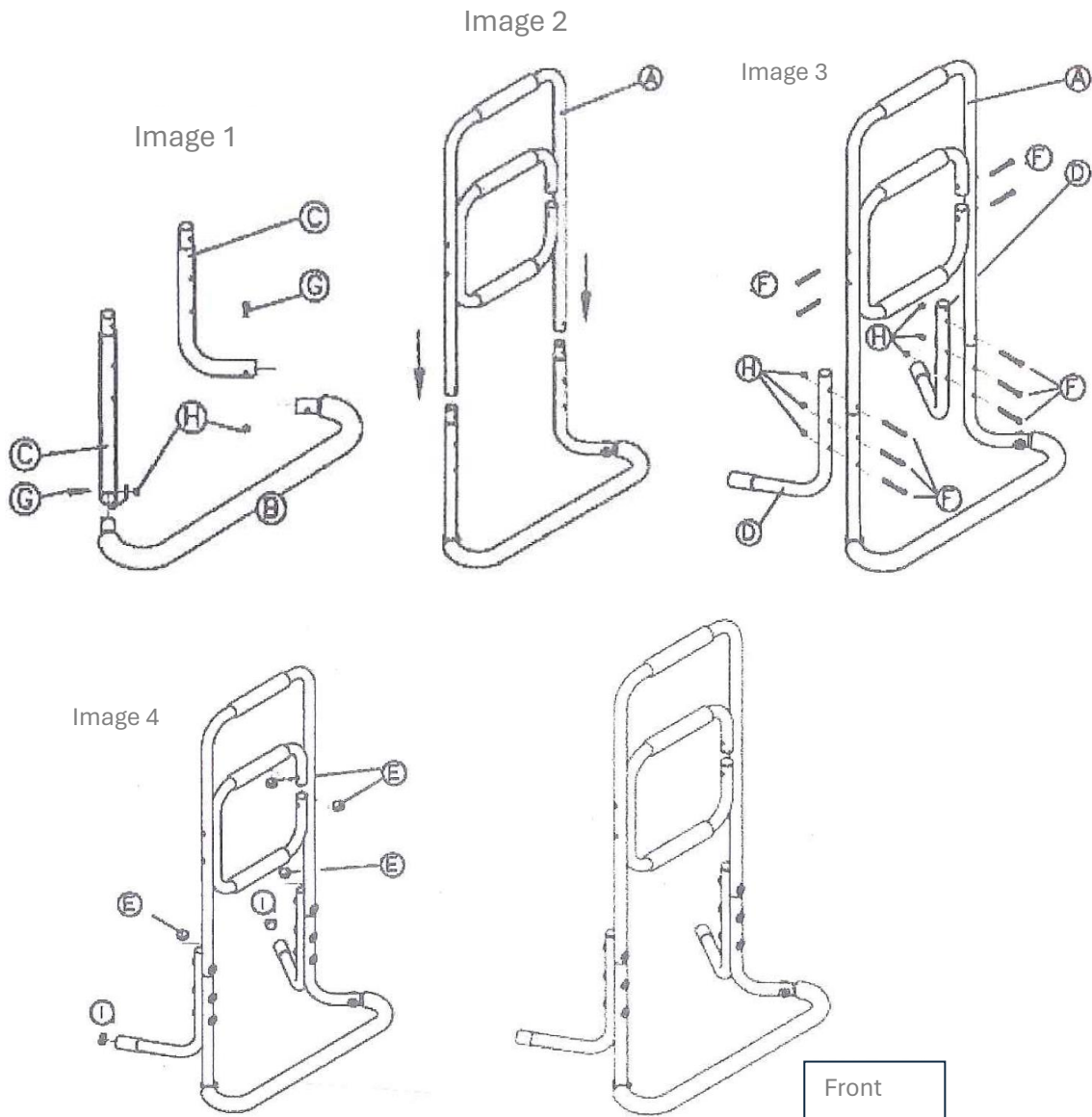Max Weight capacity of 350Lbs.

CARE AND CLEANING INSTRUCTIONS:

- Do NOT use abrasive cleaners or sharp utensils to clean this product.
- Store the item in a cool, dry place.
- Wipe with a clean dry cloth.

DISPOSAL

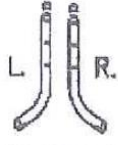
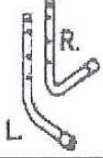
- Coopers of Stortford use recyclable or recycled packaging where possible.
- Please dispose of all packaging, paper, cartons in accordance with your local authority recycling regulations.
- At the end of the product's lifespan please check with your local council authorised household waste recycling centre for disposal.

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UK: The Enterprise Department Ltd.