



Dimensions

77cm *22.5cm

Rated voltage: 5V

Input: DC5V, 2A

- HEAT THERAPY / 3 TEMP LEVELS 45-67 degrees / USB CHARGED / Unisex / Adjustable
- The thick neoprene material is an excellent insulator for retaining heat, thereby improving blood circulation around the region and accelerating general recovery
- **Heated knee wrap** can be used on thigh, knee, arm and calf etc to provide pain relief
- Keep **Knee Warm**.
- Stimulate Blood Circulation. Alleviate Muscle Soreness. Relieve **Knee** Joint Pains. Penetrate Therapy for Recovery.
- Used to relieve muscular aches and joint pains associated with overexertion, strains and sprains as well as pains
- Heat Therapy Knee Brace Wrap Support Thermotherapy Heating Pad Hot Compress for Knee Sprains, Cramps, Joint Pain, Arthritis Pain Relief Fits Men and Women

ASSEMBLY INSTRUCTIONS / INSTRUCTIONS FOR USE:

Unpack the components carefully. Please check all parts have been removed from the packaging.

Power connection:

Insert the USB receiver into the energizing port of the kneepad and plug the other end into the power port.

This product is suitable for general USB interface line, which can be connected to mobile power supply, computer, ordinary and vehicle-mounted power supply, etc. The knee wrap needs to be kept plugged into USB port when being used at all times.

Temperature control operation:

1. Start

After the power supply is connected, the middle power supply button will turn on yellow/blue/green colour. Press the power button for 3seconds to start the machine, and it shows the red light, press the button again, it shows the blue light, once again, it shows the green lights. and it will be on the boundary of the medium temperature range.

Head Office, 11 Bridge Street,
Bishop's Stortford CM23 2JU
Tel: 0844 482 4400*
Fax: 01279 756 595
www.coopersofstortford.co.uk

Coopers
OF STORTFORD

2. Turn off

Press the power button for 3seconds, the light will go out and unplug the cord.

After switched on, press the button to adjust the temperature, there are 3 temperature levels.



Red 63-67 °C



Blue 52-56 °C



Green 41-45 °C

Wear method:

Tie kneepad onto knee, adjust it according to kneepad circumference, fix the buckle of kneepad two ends to stick well.

Attention:

1. It is recommended to wear it for about 30minutes every day. Choose a comfortable temperature, and extend the time according to your own physical condition and usage habits.
2. Keep away from pregnant woman and infants.
3. Keep the port and plug dry always.
4. The wrap can be used directly on the skin but should be removed if discomfort is experienced.

CARE AND CLEANING INSTRUCTIONS:

- If cleaning is required, remove the power cord out of the kneepad. Hand washing is strongly recommended and the water temperature should not exceed 40 °C.
- Air dry and do not use dryer.

DISPOSAL

- Coopers of Stortford use recyclable or recycled packaging where possible.
- Please dispose of all packaging, paper, cartons, packaging in accordance with your local recycling regulations.
- At the end of the product's lifespan please check with your local council authorised household waste recycling centre for disposal.

