

Hand Therapy Heat Massager G608

Instructions for Use

Please read and retain these instructions for future reference

v003: 11/07/22



Technical details

Rated output: 5VDC, 2A

Charging time: approx. 3 hours

Important

For best results, the massager needs to be fully charged. Also ensure your hand is in the correct position.

SAFETY INSTRUCTIONS:






This symbol is used to alert users to a specific operating procedure that must not be performed.

Safety caution

To reduce the risk of injury, loss of life, electric shock, fire, malfunction, and damage to equipment or property, always observe the following safety precautions.

Explanation of symbols

The following symbols are used to classify and describe the level of hazard, injury, and property damage caused when the denotation is disregarded and improper use is performed.

 DANGER	Denotes a potential hazard that will result in serious injury or death.
 WARNING	Denotes a potential hazard that could result in serious injury or death.
 CAUTION	Denotes a hazard that could result in minor injury.

The following symbols are used to classify and describe the type of instructions to be observed.



This symbol is used to alert user to a specific operating procedure that must be followed in order to operate the unit safely.



Do not use in a way exceeding the rating of the household outlet or wiring. Exceeding the rating by connecting too many plugs to one household outlet may cause fire due to overheating.



Pregnant women should not use this appliance. Hormone imbalance may cause skin problems.

COOPERS
OF STORTFORD

UK Head Office, 11 Bridge Street, Bishop's Stortford, CM23 2JU

Tel: 0844 4824400*, Fax: 01279 756595

www.coopersofstortford.co.uk





EU address: D.S.B. LD, 160 Bd. de Fourmies, 59100 Roubaix, France

Tel: (+44) 1279 701269







*UK Calls cost around 7 pence per minute plus your telephone company's network access.

Please check with your telephone supplier.

WARNING

-  **This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety. Children should be supervised to ensure that they do not play with the appliance.**
- Otherwise it may cause an accident or injury.
-  **The supply cord cannot be replaced. If the cord is damaged, the AC adaptor should be scrapped.**
- Otherwise it may cause an accident or injury.
-  **Regularly clean the power plug and the appliance plug to prevent dust from accumulating.**
- Otherwise it may cause fire due to insulation failure caused by humidity.
Disconnect the adaptor and wipe with a dry cloth.
-  **Properly dispose the package of the appliance, keep the package away from infants.**
- Otherwise it may cause suffocation of infants from happening.

CAUTION

-  **Do not wrap the cord tightly around the adaptor when storing.**
- Otherwise it may cause the wire in the cord to break with the load, and may cause fire due to a short circuit.
-  **Do not allow pins or trash to adhere to the power plug or the appliance plug.**
- Otherwise it may cause electric shock or fire due to a short circuit.
-  **Do not engage in any other activities while wearing the main unit.**
- Otherwise it may lead to unexpected accidents.
-  **Do not drop or subject to shock.**
- Otherwise it may cause injury.
-  **Disconnect the adaptor from the household outlet when not charging.**
- Otherwise it may cause electric shock or fire due to electrical leakage resulting from insulation deterioration.
-  **Disconnect the adaptor or the appliance plug by holding onto the adaptor or appliance plug instead of the cord.**
- Disconnecting by holding onto the cord may cause electric shock or injury.



CAUTION



The appliance should be cleaned to avoid the build up of grease and other residues.



The appliance has a heated surface.
Persons insensitive to heat must be careful when using the appliance.



Do not lie or sit down on the massager.



Do not cause impact to the massager.



Do not cover yourself with a blanket or quilt while using the massager.



Do not use while sleeping.


Hand Therapy Heat Massager G608

Instructions for Use

Please read and retain these instructions for future reference

v003: 11/07/22

Part name

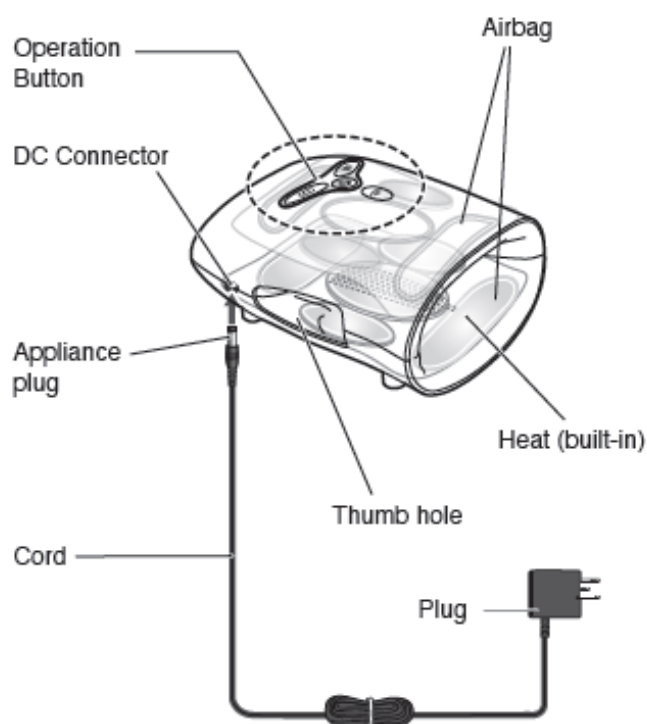

COOPERS
OF STORTFORD

UK Head Office, 11 Bridge Street, Bishop's Stortford, CM23 2JU

Tel: 0844 4824400*, Fax: 01279 756595

www.coopersofstortford.co.uk

EU address: D.S.B. LD, 160 Bd. de Fourmies, 59100 Roubaix, France

Tel: (+44) 1279 701269

*UK Calls cost around 7 pence per minute plus your telephone company's network access.

Please check with your telephone supplier.

Before using the unit

⚠ ATTENTION

- Please check the massager's fabric cover for damage or tear before use. Do not use it if the fabric is torn.
- Please ensure that the operation buttons and
- If you have not used the unit for some time, please read the instruction manual carefully again, make sure the machine is working normal




Inner cover of massager
Please check if the fabric
cover is damaged.

How to recharge

1 Connect the power adapter interface and products

Connect the power adapter interface and products.

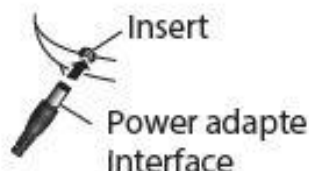

2 Plug the power adapter charging head into the socket

To prevent electric shock, please plug into the socket accurately and safely.

- ※ Start charging, the power button flashes red.
- ※ A full charging takes about 3 hours for a completely discharged battery.




3 Check if the charging light is on.

- ※ The battery may discharge automatically if it's idle for a long time.
- ※ You can use the massager with any functions during the device is being charged.

How to recharge

- ✘ During charging, the power button indicator will glow red.
- ✘ Charging completed when the indicator light changes from flashing red to long red light.

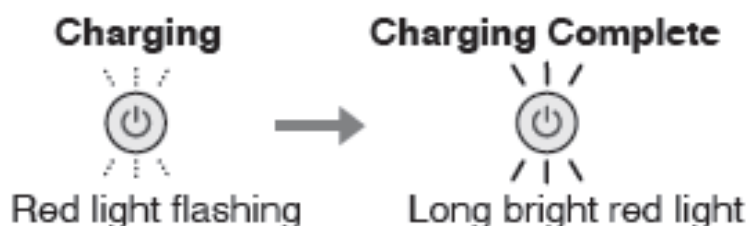

It is fully charged in about 3 hours and can be used for about 1.5 hours.

- ✘ If there is still some electricity left in the battery, the charging time will be shortened accordingly.

Charging method is different, but usually can be charged about 200 times.

- ✘ If it is charged in a high temperature environment, the product protection device will work and the charging may stop halfway.

- 4 When charging is completed, please disconnect the power adapter and charging plug.**

How to use



ATTENTION

Please use the slightly lower level massage function when starting up the appliance.
Stop using it if you feel anything abnormal.

5 Insert your hand into the product


- Open your palm and put into the appliance, place your thumb and other four fingers like below images.



4 fingers out from here



Thumbs in here

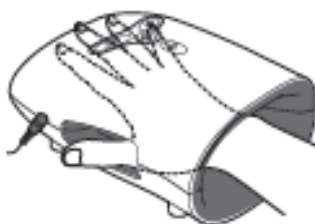

Fully insert

COOPERS
OF STORTFORD

UK Head Office, 11 Bridge Street, Bishop's Stortford, CM23 2JU

Tel: 0844 4824400*, Fax: 01279 756595

www.coopersofstortford.co.uk

EU address: D.S.B. LD, 160 Bd. de Fourmies, 59100 Roubaix, France

Tel: (+44) 1279 701269

*UK Calls cost around 7 pence per minute plus your telephone company's network access.

Please check with your telephone supplier.

How to use



ATTENTION

Usage examples

Do not pull out your hand during use.
Please turn off the appliance by pressing the power button if in emergency.



Left hand



Right hand



Heat Button

White light on when heat function is in operation.




Vibration Button

White light on when vibration function is in operation.

Air Pressure intensity button

weak: 1 light
medium: 2 lights
strong: 3 lights

✕ Default slightly lower level when turn on.

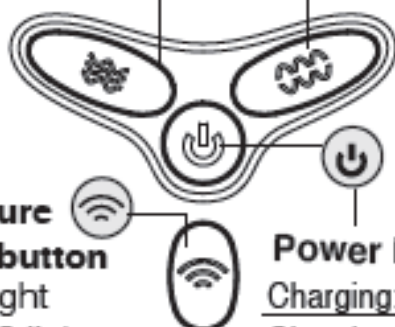

Power Button

Charging: red light flashes
Charging completed:red light on
Massage mode in operation:
green light on
Low battery: green light flashes

COOPERS
OF STORTFORD

UK Head Office, 11 Bridge Street, Bishop's Stortford, CM23 2JU

Tel: 0844 4824400*, Fax: 01279 756595

www.coopersofstortford.co.uk

EU address: D.S.B. LD, 160 Bd. de Fourmies, 59100 Roubaix, France

Tel: (+44) 1279 701269

*UK Calls cost around 7 pence per minute plus your telephone company's network access.

Please check with your telephone supplier.

How to use

6 Press the power to start massage




Power Button

Press the power button to start the 「air pressure-weak」 timing for 10 minutes.

Other function buttons can be optional. Turn off the appliance by pressing the power button once more.

※ The light of power button: Green light on when fully charged. Green light flashes slowly in low battery condition.

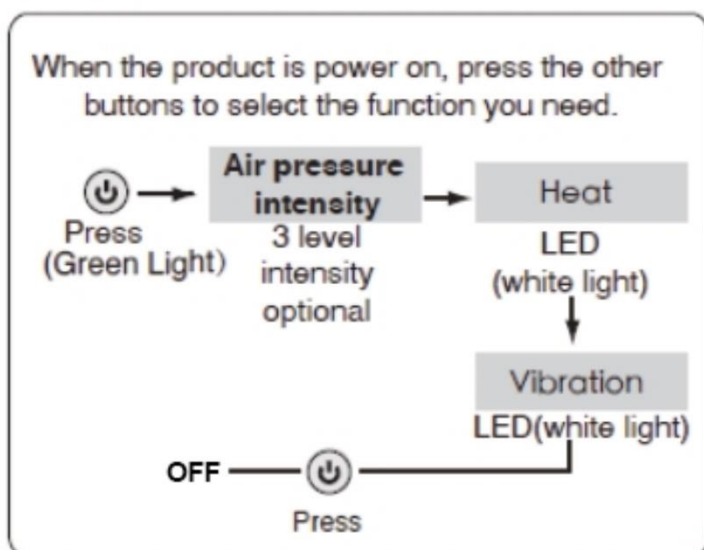


How to use




Air Pressure Intensity Button

Press this button to switch among weak→medium→strong mode in turn during the massage.

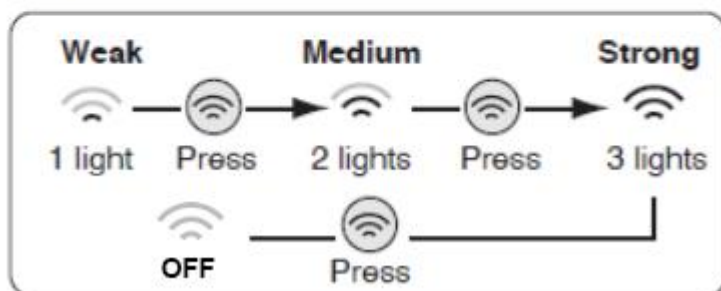

Heat Function

Press the heating button in the power on state,
(Heating turn on) ↔ (Heating turn off)
switch in sequence

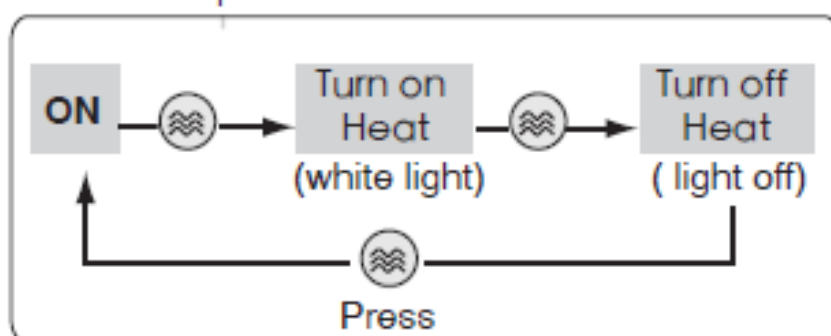


How to use



Vibration Button

Press the vibrating button in the power on state,
(Vibrating turn on) \longleftrightarrow (Vibrating turn off)

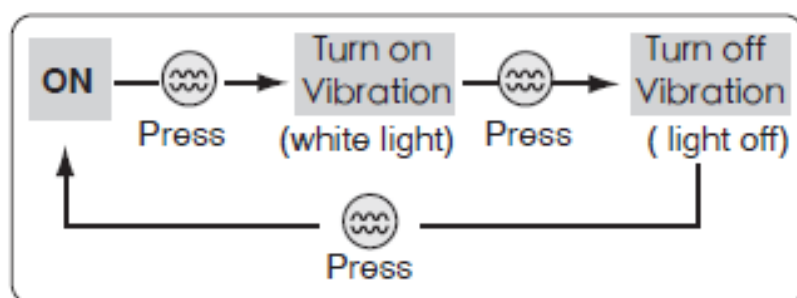

- ✧ Before the product is turned on, other buttons are invalid, and it will shut off automatically when time is up .
- ✧ The working time is 10 minutes when product is turned on.

Removing the built-in rechargeable battery

- This product uses a rechargeable lithiumion battery. Remove the built-in rechargeable battery before disposing of this product.



WARNING

Do not disassemble unless the product is discarded.

How to remove the battery

- The following breakdown drawings are used for discarding product, not for repairing.

1. Remove the AC adapter and press the power button to check if it works


If the battery still support the appliance working, operate till the power off.

Removing the built-in rechargeable battery

2. Turn the product over and remove the 4 rubber feet and 4 rubber caps which covering the screws.

Rubber cap


Rubber foot pad

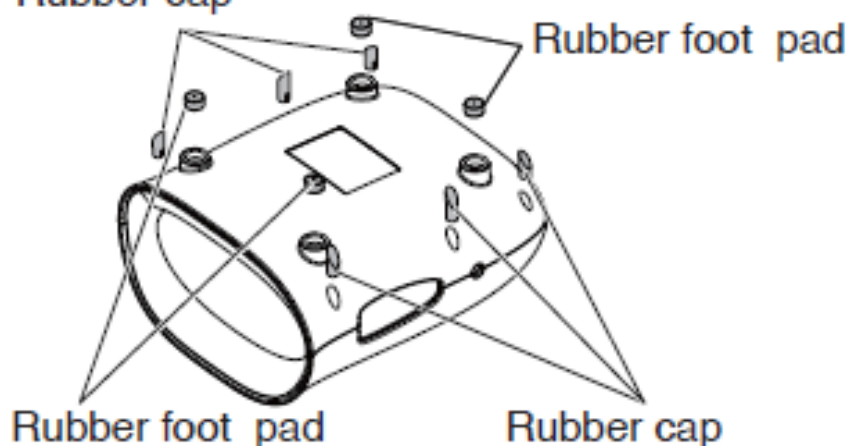

3. Use a  screwdriver to remove the screw

Screw (5 pieces)

Screw (5 pieces)

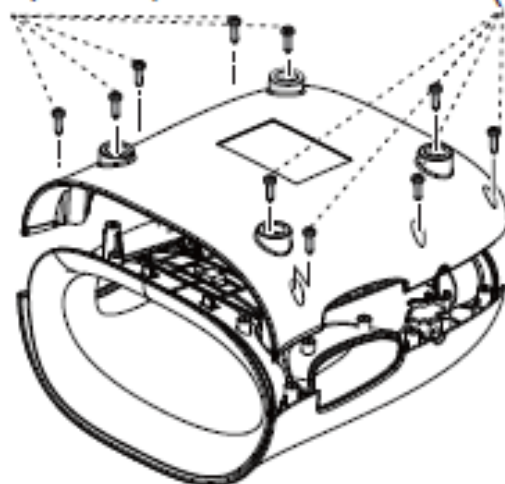

COOPERS
OF STORTFORD

UK Head Office, 11 Bridge Street, Bishop's Stortford, CM23 2JU

Tel: 0844 4824400*, Fax: 01279 756595

www.coopersofstortford.co.uk

EU address: D.S.B. LD, 160 Bd. de Fourmies, 59100 Roubaix, France


Tel: (+44) 1279 701269

*UK Calls cost around 7 pence per minute plus your telephone company's network access.

Please check with your telephone supplier.

Removing the built-in rechargeable battery

4. Turn the product over, follow along the battery cable and find the cable terminal, then unplug it.
5. Remove the battery

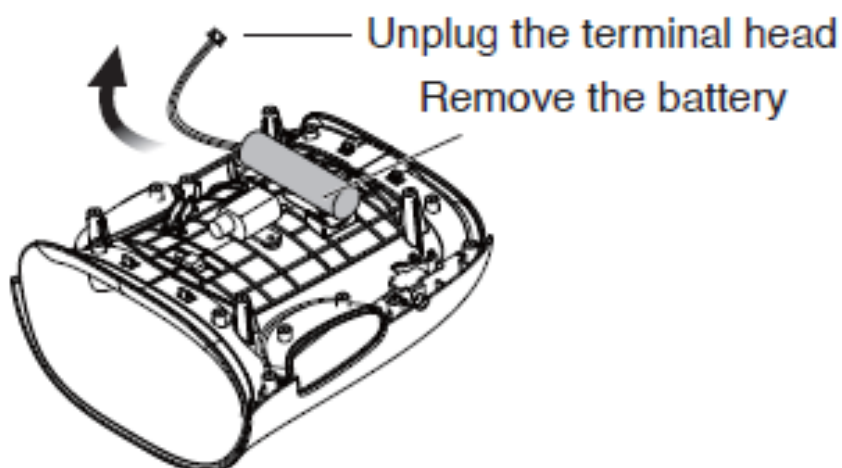

Be sure to remove the power adapter from the product before disassembling.

Note Please keep the battery away from short-circuit carefully.

Troubleshooting

Problem	Possible cause	Action
The appliance does not operate even the "power" switch is pressed.	▶ The main unit and AC adaptor are connected to a household outlet.	▶ Remove the AC adaptor from the main unit.
	▶ It is not charged	▶ Recharge with the AC adaptor. (See page 7.)
	▶ The "power" switch is not firmly pressed.	▶ Press the "power" button firmly to its place.
The charging indicator lamp does not go on/goes out.	▶ The appliance is fully charged. This does not indicate a problem. (See page 7.)	
	▶ The adaptor is not properly inserted into the household outlet.	▶ Properly insert the adaptor into the household outlet.
	▶ The appliance plug is not firmly inserted into the appliance socket.	▶ Firmly insert the appliance plug into the appliance socket.
The appliance can be used only for a few minutes even after charging it.	▶ Battery life is exhausted (approximately 5 years).	▶ Consult a retailer or an authorized service center. (Do not change the battery yourself. Doing so may damage the appliance.)
The adaptor is abnormally hot.	▶ It is loosely inserted into the household outlet.	▶ Use a household outlet that is not loose.
The main unit does not heat up.	▶ The appliance may not work beyond the recommended temperature.	▶ Use under the recommended temperature of 15 °C - 35 °C
The appliance stops during use.	▶ The battery has run down.	▶ Recharge with the AC adaptor.
	▶ The appliance automatically stops after approximately 10 minutes.	
	▶ This does not indicate a problem.	

How to clean

Cleaning

Before you clean, please make sure the power is switched off, and the power plug has been pulled out from the socket. Please use dry cloth to clean the product.


Maintenance

Keep the product clean and far from the wet environment.

Clean when you feel cleaning is necessary.

Cleaning the main unit

- Wipe away dirt with a cloth soaked in soapy water, wring out the cloth before wiping.
- Do not use alcohol, paint thinner, benzine, or nail polish remover. Using these may damage the appliance or cause cracks or discoloration in the components.

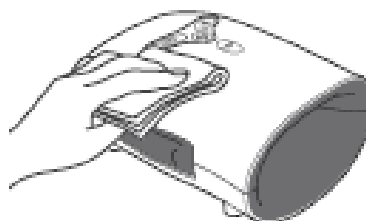

BATTERIES

- Keep batteries away from children and pets.
- Do not mix used and new batteries.
- Remove the batteries if left for long periods.
- Never dispose of batteries in a fire.
- Battery disposal, spent or expired batteries must be properly disposed of and recycled in compliance with local regulations. For detailed information, contact your local authority.

**COOPERS
OF STORTFORD**

UK Head Office, 11 Bridge Street, Bishop's Stortford, CM23 2JU

Tel: 0844 4824400*, Fax: 01279 756595

www.coopersofstortford.co.uk

EU address: D.S.B. LD, 160 Bd. de Fourmies, 59100 Roubaix, France

Tel: (+44) 1279 701269

*UK Calls cost around 7 pence per minute plus your telephone company's network access.

Please check with your telephone supplier.

Hand Therapy Heat Massager G608

Instructions for Use

Please read and retain these instructions for future reference

v003: 11/07/22

- Follow the battery manufacturer's safety, usage, and disposal instructions.

COOPERS
OF STORTFORD

UK Head Office, 11 Bridge Street, Bishop's Stortford, CM23 2JU

Tel: 0844 4824400*, Fax: 01279 756595

www.coopersofstortford.co.uk

EU address: D.S.B. LD, 160 Bd. de Fourmies, 59100 Roubaix, France

Tel: (+44) 1279 701269

*UK Calls cost around 7 pence per minute plus your telephone company's network access.

Please check with your telephone supplier.

PLUG WIRING (UK & IRELAND)

This appliance is supplied with a mains adaptor/charger with BS1363 plug pins.

DO NOT use if the socket outlets in your home or office are not suitable for this product's mains adaptor/charger plug pins. Attempts to insert the mains adaptor/charger into the wrong socket is likely to cause electric hazard or damage.

DO NOT use with any other mains transformer/charger than the one that is supplied with this product.

DISPOSAL

- Dispose of all packaging, paper, cartons, plastic and plastic bags in accordance with your local recycling regulations.
- At the end of the product's lifespan please dispose of it at an authorised household WEEE waste recycling centre.

UK
CA



COOPERS
OF STORTFORD

UK Head Office, 11 Bridge Street, Bishop's Stortford, CM23 2JU

Tel: 0844 4824400*, Fax: 01279 756595

www.coopersofstortford.co.uk

EU address: D.S.B. LD, 160 Bd. de Fourmies, 59100 Roubaix, France

Tel: (+44) 1279 701269

*UK Calls cost around 7 pence per minute plus your telephone company's network access.

Please check with your telephone supplier.