

Informative Easy Read Clock

H001 v01: 10/07/20

Please read and retain these instructions for future reference

vivadia▶



Dimensions

21.5cm x17cm x2.4cm

Screen size: 16cm x12cm

- LARGE EASY READ DIGITS
- 16CM WIDE, ANTI-GLARE, LED FULL COLOUR DISPLAY SCREEN
- CLEARLY DISPLAYS THE DAY, DATE & TIME AS WELL AS WHETHER IT IS MORNING, AFTERNOON OR EVENING
- USE FREE STANDING OR WALL MOUNTED
- EASY SET UP
- 3 CHOICES OF MAIN CLOCK DISPLAY
- AUTO NIGHT DIMMER AS WELL AS MANUAL SCREEN BRIGHTNESS ADJUSTMENT
- HIGH VIEWING ANGLE
- 5 ALARM CHOICES AND 3 GROUPS OF MEDICATION REMINDER ALARMS

ASSEMBLY INSTRUCTIONS / INSTRUCTIONS FOR USE:

Unpack the components carefully. Please check all parts have been removed from the packaging.

Contents


8" day clock x 1; 5V2A power adapter x 1; User Manual x 1



*Calls cost 7 pence per minute plus your telephone company's network access charge

The Enterprise Department
Head Office: 11 Bridge Street
Bishop's Stortford, CM23 2JU
Tel: 0844 482 4400*
Fax: 01279 756 595
www.coopersofstortford.co.uk

Parts

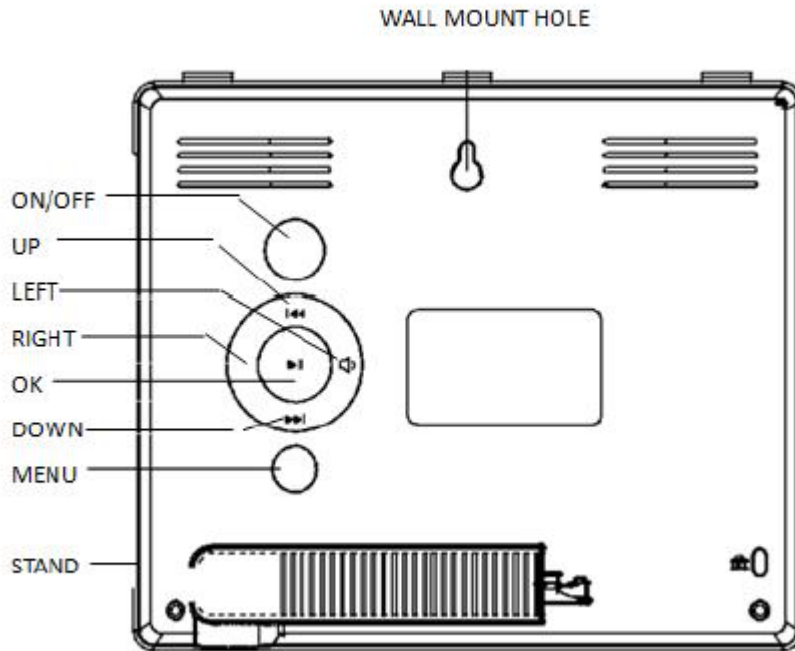

Settings:

Press the **MENU** button to display the on screen menu:

Please Note: Changes can only be made to the line when underlined the digits and in yellow font. Use **UP** or **DOWN** button to adjust the digits. Press **LEFT** or **RIGHT** buttons to choose the digits you would like to adjust. Press **OK** to save and **MENU** to exit.

Language Set	<	English	>
Time Set	<	04:47 PM	>
Date Set	<	08-12-2017	>
Time Mode	<	12Hours	>
Date Mode	<	Day-Month-Year	>
Auto Dimming	<	Auto	>
Volume Set	<	28	>
Alarm Set			>
Medicine reminder			>
Factor Reset	<	V5.1	>

Line 1- Language Set:

Press "**MENU**" to enter into the settings, Press **UP** or **DOWN** to the line you would like to change ,press **OK** to switch to the languages you prefer, Press **MENU** to exit

Line 2- Time Set:

Goes to the line of **Time Set**, press **"OK"** to underline the hour or minute, Press **UP** or **DOWN** to adjust ,then press **"OK"** to save ,and press **MENU** to exit.

Line3 – Date Set:

Goes to the line of **Date Set**, press **"OK"** to underline the day, date or year, Press **UP** or **DOWN** to adjust,then press **"OK"** to save ,and press **MENU** to exit.

Line4 - Time Mode:

Goes to the line of **Time Mode**, press **OK** to switch to **12 Hour** or **24 Hour** you prefer, Press **MENU** to exit.

Line5 - Date Mode: <Day-Month-Year>or<Month-Day-Year> or<Year-Month-Day>

Goes to the line of **Date Mode**, press **OK** to switch to the date mode you prefer, Press **MENU** to exit.

Line6 – Auto Dimming

Goes to the line of **Auto Dimming**, press **OK** to switch to the one of the four modes (**auto, off, mid, low**) you prefer, Press **MENU** to exit.

Auto: Normal brightness starts from 7:00am to 7:59pm, screen dimmed to low brightness from 8:00pm to 6:59am)

Off: Always on normal brightness

Line7 – Volume Set:

Goes to the line of **Volume Set**, press **OK** to switch to the volume level from 00 to 30 you prefer, Press **MENU** to exit.

Line8 - Alarm Set:

Goes to the line of **Alarm Set**, press **OK** to the five groups of alarm clocks settings, press **UP** or **DOWN** to choose the **Time, Date** or **Mode** settings. Press **MENU** to exit.

Time: Same operation as you do **time set**

Date: Press **OK** to switch to one of the four frequencies you prefer, **Monday to Friday/Saturday to Sunday/Monday to Sunday/Off**

Mode: Press **OK** to switch between **once** and **repeat**

Line9 – Medication Reminder

Goes to the line of **Medication Reminder**, press **OK** to the 3 groups of medication reminders settings, press **UP** or **DOWN** to choose the **Time, Status** or **Mode** settings.

Press **MENU** to exit.

Time: Same operation as you do **time set**

Status: Press **OK** to switch between **On** and **Off**

Mode: Press **OK** to switch between **once** and **repeat**

Line10 – Factory reset

Goes to the line of **Factory reset**, press **OK** to choose reset to factory default settings or not, press **MENU** to exit.

Default Periods of the Day:

- 6:00-11:59 Morning
- 12:00-17:59 Afternoon
- 18:00-23:59 Evening
- 24:00-5:59 Night

Tips: USB port for factory software upgrading only!

PLUG WIRING (UK & IRELAND)

This appliance is supplied with a mains adaptor/charger with BS1363 plug pins.

DO NOT use if the socket outlets in your home or office are not suitable for this product's mains adaptor/charger plug pins. Attempts to insert the mains adaptor/charger into the wrong socket is likely to cause electric hazard or damage.

DO NOT use with any other mains transformer/charger than the one that is supplied with this product.


CARE AND CLEANING INSTRUCTIONS:

- Do NOT immerse in water.
- Do NOT use abrasive cleaners or sharp utensils to clean this appliance.
- Store the item in a cool, dry place.
- Wipe with a clean dry cloth.

DISPOSAL

- Coopers of Stortford use recyclable or recycled packaging where possible.
- Please dispose of all packaging, paper, cartons, packaging in accordance with your local recycling regulations.
- Plastics, polybags – this is made from the following recyclable plastic.



Code & Symbol	 OTHER
Type of Plastic	ABS
Commonly used for	CD's, crisp packets, various flexible packaging, baby bottles, sunglass lenses
Notes	Not often or widely recycled

- At the end of the product's lifespan please check with your local council authorised household waste recycling centre for disposal.

