

## Dimensions

15cm x10cm

- Comfort straps to ease the pressure of bunions and overcrowding
- Set of 12
- Soothe bunion pain
- Adhesive strap with cushion on centre point
- Can be used in shoes or while barefoot
- Easy wearing
- Helps align toes to prevent crowding, overlap and friction



The protective patches are to be attached crosswise to the big toe to protect it against friction and pressure points. They are also comfortable to wear in shoes and do not slip.

## User Instructions


- Use on dry, clean skin.
- Peel off protective film and put on as shown on the images. Single use. Do not leave on for more than 24 hours.

**WARNING! Do not use on a wound. Do not use on mucous membranes. Stop use in case of skin problem and consult a specialist. If you have diabetes, make sure that the product does not hamper your blood circulation and stop use immediately if discomfort.**

## DISPOSAL

- Coopers of Stortford use recyclable or recycled packaging where possible.
- Please dispose of all packaging, paper, cartons, packaging in accordance with your local recycling regulations.
- Plastics, polybags – this is made from the following recyclable plastic.



Code & Symbol	 OTHER
Type of Plastic	EVA
Commonly used for	CD's, crisp packets, various flexible packaging, baby bottles, sunglass lenses
Notes	Not often or widely recycled

- At the end of the product's lifespan please check with your local council authorised household waste recycling centre for disposal.