

Magnetic Posture Correction Reminder Brace

H604 v02: 17/06/22

Please read and retain these instructions for future reference

vivadia▶



- INVISIBLE UNDER CLOTHES
- INSTANTLY IMPROVES POSTURE
- IMMEDIATELY REDUCES SLOUCHING & IMPROVES POOR POSTURE WITH EASILY ADJUSTABLE STRAPS
- SECURES YOUR SHOULDERS, NECK & BACK IN THEIR NATURAL POSITION BY PUTTING PRESSURE ON YOUR MUSCLES TO KEEP THEM CORRECTLY ALIGNED.
- MAGNETS OFFER PAIN RELIEF AFTER YEARS OF BAD POSTURE
- ADJUSTABLE ONE SIZE
- **SHOLDER STRAPS 5.2CM, LENGTH 19CM ARM CIRCUMFERENCE 82CM**



WARNING!

This item contains small magnets. Magnets can interfere with pacemakers and ICDs. Keep away from pacemakers and ICDs to avoid interference to safe operation.

This is not a toy. Keep away from children and pets.

Magnets sticking together or becoming attached to a metallic object inside the human body can cause serious or fatal injury - seek immediate medical help if magnets are swallowed or inhaled.



CAUTION

- NEVER USE THE MAGNETIC POSTURE CORRECTION REMINDER BRACE IF IT IS NOT WORKING PROPERLY OR IF THERE ARE DAMAGED OR WORN PARTS.
- PLEASE READ INSTRUCTIONS COMPLETELY BEFORE USING THIS PRODUCT.
- CONSULT YOUR GP BEFORE STARTING ANY TRAINING.
- START SLOWLY AND PROGRESS SENSIBLY.
- DO NOT OVEREXERT YOURSELF WITH THIS OR ANY OTHER EXERCISE PROGRAM.
- FOR NON-COMMERCIAL USE ONLY.
- USE THIS EQUIPMENT ONLY FOR THE INTENDED USE AS DESCRIBED BY THE MANUFACTURER.
- HAVE PLENTY OF CLEARANCE SPACE. KEEP CHILDREN, PETS, FURNITURE AND OTHER OBJECT OUT OF THE WAY WHEN USING YOUR EQUIPMENT.
- THIS PRODUCT IS FOR ADULT USE ONLY. PLEASE KEEP AWAY FROM CHILDREN DURING USE AND WHEN EQUIPMENT IS UNATTENDED.
- FAILURE TO READ AND FOLLOW INSTRUCTIONS STATED IN THIS MANUAL MAY RESULT IN SERIOUS INJURY.

**The Enterprise
Department**

UK Head Office, 11 Bridge Street, Bishop's Stortford, CM23 2JU

Tel: 0844 4824400*, Fax: 01279 756595

www.coopersofstortford.co.uk

EU address: D.S.B. LD, 160 Bd. de Fourmies, 59100 Roubaix, France

Tel: (+44) 1279 701269

*UK Calls cost around 7 pence per minute plus your telephone company's network access.
Please check with your telephone supplier.


Magnetic Posture Correction Reminder Brace H604 Instructions for Use

Please read and retain these instructions for future reference

v002: 17/06/22


Step 1

Loosen the hook and loop fasteners to their maximum length. Adjust the shoulder pads.

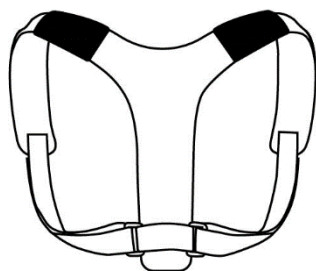


Step 2

Slip arms through loops like backpack straps.

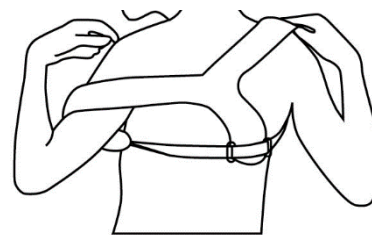


Step 3

Adjust front tightening straps until the desired effect is reached.

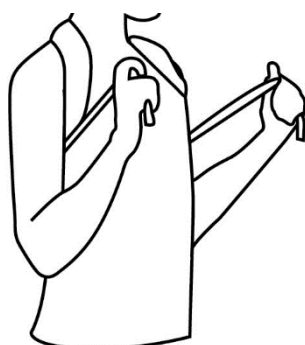

Step 4

Wear the corrector for **30 minutes for the first few days**. Then gradually increase time by 20 minutes per day.

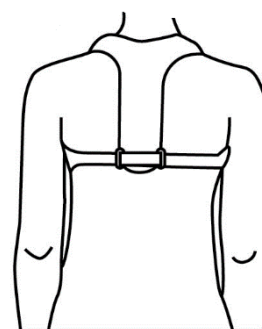

Cleaning and Maintenance

- Hand wash in cold water (30°C).
- Do not bleach.
- Air dry.

DISPOSAL

- Coopers of Stortford use recyclable or recycled packaging where possible.
- Please dispose of all packaging, paper, cartons, packaging in accordance with your local recycling regulations.
- At the end of the product's lifespan please check with your local council authorised household waste recycling centre for disposal.



**The Enterprise
Department**

UK Head Office, 11 Bridge Street, Bishop's Stortford, CM23 2JU

Tel: 0844 4824400*, Fax: 01279 756595

www.coopersofstortford.co.uk

EU address: D.S.B. LD, 160 Bd. de Fourmies, 59100 Roubaix, France

Tel: (+44) 1279 701269

*UK Calls cost around 7 pence per minute plus your telephone company's network access.
Please check with your telephone supplier.