

Inflatable Neck Support

H726 v01: 07/05/21

Please read and retain these instructions for future reference

vivadia



Dimensions

(H) 20 x (W) 21 (D) 24cm

- INFLATABLE NECK SUPPORT, INCLUDES PUMP
- DESIGNED TO HELP RELAX YOUR NECK AND SHOULDERS TO HELP REDUCE HEADACHES, NECK, SHOULDER AND HAND PAIN WITH THE NECK SUPPORT AND TRACTION COLLAR.
- IMPROVE POSTURE BY STRETCHING AND RELAXING NECK AND SHOULDER MUSCLES
- 3 LAYER DESIGN
- FLOCKED TOP AND BOTTOM ENSURES A SOFT AND COMFORTABLE FIT
- INFLATE TO REACH THE SUPPORT YOU NEED
- SIMPLE INFLATABLE DESIGN WITH HAND PUMP
- RELIEF FOR NECK AND SHOULDER MUSCLES
- HOOK AND LOOP FASTENING TOP AND BOTTOM

ASSEMBLY INSTRUCTIONS / INSTRUCTIONS FOR USE:

Unpack the components carefully. Please check all parts have been removed from the packaging.




Feature:

Breathable comfortable, Portable.

Advantages of 3 Layers INFLATABLE NECK SUPPORT:


1. Anatomically corrects traction safely.
2. Comfortably supports head and neck.
3. Exerts force evenly and gently.
4. Supports normal curvature of cervical spine.
5. Gentle traction release.
6. Easy to use and convenient for the elderly.
7. Light and portable for traveling.

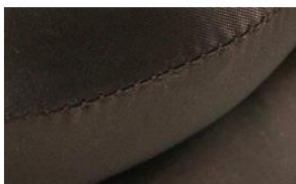

Blue Plastic Air Control Valve



Tube

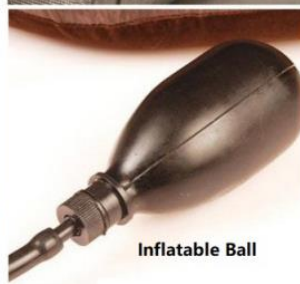

Inflatable Ball

Inflation Procedure:



1. : Put on the neck support with the assistance of a mirror, adjust the size of the neck support, fasten the 2 hook and loop straps.



2. : Move the controlling valve to the top to ensure smooth inflation



3. : Pump the traction with air ball to a comfortable level.



4. : Move the controlling valve to the bottom to prevent air escaping.



5. : Enjoy the comfort



CARE AND CLEANING INSTRUCTIONS:

- Wipe with a clean dry cloth.

DISPOSAL

- Coopers of Stortford use recyclable or recycled packaging where possible.
- Please dispose of all packaging, paper, cartons, packaging in accordance with your local recycling regulations.
- At the end of the product's lifespan please check with your local council authorised household waste recycling centre for disposal.



Code & Symbol	 PVC	 PP
Type of Plastic	Polyvinyl Chloride	Polypropylene
Commonly used for	Pipes, construction material, flooring, windows, guttering, blister packs	Microwaveable food containers, yoghurt pots, butter tubs, disposable plates and cups
Notes	Not usually collected from household recycling bins, Sometimes recycled at local waste management centres.	Commonly recycled and collected from household recycling bins (packaging tape itself not recyclable but does not interfere with cardboard)