

# Pk2 Vacuum Ear Cleaners

L064 v02: 12/04/23

Please read and retain these instructions for future reference

vivadia



## Dimensions

(W) 10 x (D) 6 x (L) 13cm

Material: Plastic

- GENTLE EAR CLEANERS
- SAFELY REMOVE EAR WAX
- COMPLETE WITH THREE WASHABLE SILICONE TIPS, A FILTER AND A CLEANING BRUSH, USE IT TIME AND TIME AGAIN.

Batteries required: 2x AA (not included) (per unit)

**ALWAYS USE ALKALINE BATTERIES.**

**DO NOT USE LITHIUM BATTERIES.**

## ASSEMBLY INSTRUCTIONS / INSTRUCTIONS FOR USE:

Unpack the components carefully. Please check all parts have been removed from the packaging.

## WARNINGS

- Check with your paediatrician before using on young children
- Never use without the silicone attachments
- Wash thoroughly before and after each use

## User Instructions:

1. Insert 2x AA batteries (not included) as shown:
2. Wash your hands thoroughly
3. Slide a silicone attachment (larger diameter in first) onto the small tube located in the front of the ear cleaner. There are four different colour silicone attachments for multiple users
4. Slide the ON/OFF switch to the ON position
5. Insert the tube with the silicone attachment into your ear, slightly turn the ear cleaner to help draw out the wax .



## How To Clean:

- Place silicone attachments in warm soapy water
- Unsnap the front piece with the small tube from the main unit and place in warm soapy water
- Remove the screen (by pulling away from the main unit) and place in warm soapy water
- Scrub each piece using the small brush provided to remove any debris
- Dry each piece thoroughly using a dry cloth

## BATTERIES

- Keep batteries away from children and pets.
- Do not mix used and new batteries.
- Remove the batteries if left for long periods.
- Never dispose of batteries in a fire.

**The Enterprise  
Department**


UK Head Office, 11 Bridge Street, Bishop's Stortford, CM23 2JU  
Tel: 0330 331 0300 Fax: 01279 756595  
[www.coopersofstortford.co.uk](http://www.coopersofstortford.co.uk)  
EU address: D.S.B. LD, 160 Bd. de Fourmies, 59100 Roubaix, France  
Tel: (+44) 1279 701269

- Battery disposal, spent or expired batteries must be properly disposed of and recycled in compliance with local regulations. For detailed information, contact your local authority.
- Follow the battery manufacturer’s safety, usage, and disposal instructions.

### DISPOSAL

- Coopers of Stortford use recyclable or recycled packaging where possible.
- Please dispose of all packaging, paper, cartons, packaging in accordance with your local recycling regulations.
- At the end of the product’s lifespan please check with your local council authorised household waste recycling centre for disposal.
- Plastics, polybags – this is made from the following recyclable plastic.



Code & Symbol	
Type of Plastic	Please Specify (includes ABS, Nylon)
Commonly used for	CD’s, crisp packets, various flexible packaging, baby bottles, sunglass lenses
Notes	Not often or widely recycled

