

EYE DROP DISPENSER

Item codes: M041

Instructions for Use

v001: 13/10/23


Please read and retain these instructions for future reference

Java
wellness



INSTRUCTIONS FOR USE:

1. Before use, wash the nozzle carefully with soap and warm water and rinse well and dry.
2. Remove the screw-cap from the bottle of eye drops and remove the nozzle-cap from the eye-wash cup.
3. Screw the Eye Drop Dispenser onto the bottle, until it is tightly fixed.
4. Delicately position the Eye Drop Dispenser with the bottle above the eye, taking care that the upper part of the eye-drop per does not exceed more than $\frac{1}{4}$ of the dispenser hole.
5. Keep the eye wide open. Tilt the head backwards to a horizontal position and look at the hole in the dispenser. Press out the prescribed number of drops. If tilting the head backwards to a horizontal position is difficult, lie down.
6. Use the cap fixed to the Eye Drop Dispenser in place of the screw-on cap of the eye drop bottle.
7. The Eye Drop Dispenser may be used as a cup to wash the eyes. In this case, Insert the yellow nozzle-cap (supplied) at the base of the Eye Drop Dispenser and use it as for any other eye wash cup.
8. Use the cap supplied to cover the top of the bottle when not in use.

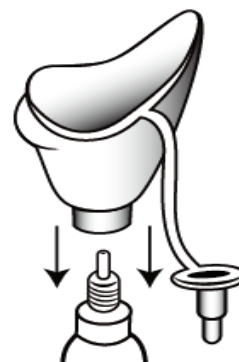


CARE AND CLEANING INSTRUCTIONS:

- Do NOT use abrasive cleaners or sharp utensils to clean this product.
- Store the item in a cool, dry place.
- Wipe with a clean dry cloth
- Do not press the rubber dispenser against the face.
- The rim of the bottle should not touch the eye.

WARNING AND CAUTIONS:

- DO NOT TOUCH THE BOTTLE RIM WITH YOUR FINGERS
- If the Eye Drop Dispenser is used at any time with a different bottle, it is essential to wash the Dispenser again with soap and warm water and rinse it thoroughly to disinfect it and prevent any risk of contamination.
- This product is to facilitate the use of eye drops already prescribed by your doctor. Please consult your doctor or pharmacist before using any medication.

DISPOSAL <ul style="list-style-type: none">• Coopers of Stortford use recyclable or recycled packaging where possible.• Please dispose of all packaging, paper, cartons in accordance with your local authority recycling regulations.• At the end of the product's lifespan please check with your local council authorised household waste recycling centre for disposal.	
--	---

CUSTOMER SERVICES:

Distributed by / Verteilt durch / Distribué par.

UK: The Enterprise Department Ltd.

11 Bridge Street, Bishops Stortford, Herts, UK, CM23 2JU

Tel: 0330 331 0300

NI/EU:

D.S.B. LD, 160 Bd. de Fourmies, 59100 Roubaix, France

Tel: (+44) 1279 701269