HEARING AMPLIFIER

Item codes:M203/215990

Instructions for Use

v001: 03/11/23

Please read and retain these instructions for future reference

java
wellness

ENGLISH INSTRUCTIONS

Power details:

1 x LR44 cell included.

Button Cell Compartment secured by screw.


Please read this instruction manual thoroughly before starting and ensure you are familiar with the operation of your new hearing amplifier.

Instructions for use:

Before use:

1. On/off switch


The correct positions of the switch are shown as below: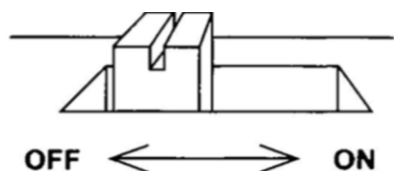

The switch should be turned to the "OFF" position when the apparatus is not in use. This will ensure longer battery life.

2. Volume control


There are 6 levels of volume control. Users are recommended to always switch the hearing amplifier on at level 1 and then adjust it to a suitable level.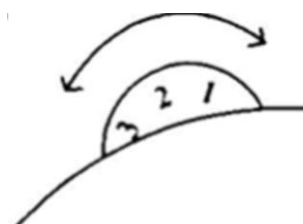

During use:

3. Battery installation

Open the battery hatch with the fingertip and inside the battery as shown below: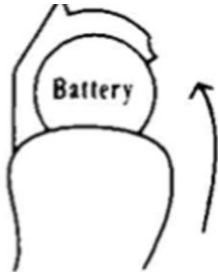

1. Turn the volume to level 1.
2. Check battery is in place.
3. Place the hearing magnifier into your ear, making sure that the earphone is right inside your ear before switching on.
4. Use the hearing amplifier only when situations require amplification of sounds e.g movies, TV.
5. Turn on the switch at the end of the hearing amplifier until you can hear an air flow noise.
6. Gradually increase the volume until it is at a suitable level.


WARNINGS AND SAFETY INSTRUCTIONS:

WHEN USING ANY APPLIANCE, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED:

- READ ALL INSTRUCTIONS BEFORE USE.
- NOT FOR COMMERCIAL USE.
- BEFORE USE, CHECK THOROUGHLY FOR ANY DEFECTS AND DO NOT USE IF DEFECTS ARE FOUND.
TAKE CARE NOT TO DROP THE PRODUCT AS HEAVY IMPACTS MAY CAUSE INTERNAL DAMAGE.
- THE USE OF ANY ACCESSORY OR ATTACHMENTS NOT RECOMMENDED BY COOPERS OF STORTFORD MAY CAUSE INJURIES AND INVALIDATE ANY WARRANTY YOU MAY HAVE.
- CAUTION THIS PRODUCT IS NOT WATERPROOF THEREFORE TAKE CARE TO PREVENT WATER FROM ENTERING HEARING AMPLIFIER.
- HEARING AMPLIFIER IS NOT A HEARING AID FOR THOSE WITH IMPAIRED HEARING. IF YOU HAVE IMPAIRED HEARING CONSULT YOUR DOCTOR.

BATTERIES:

- **CONTAINS BUTTON CELLS**

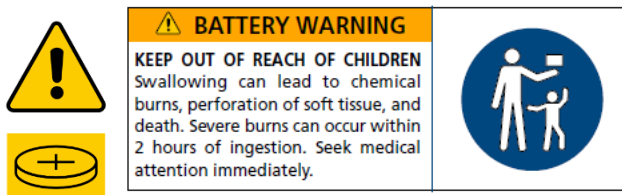BATTERY SAFETY:

- **BATTERY WARNING: KEEP OUT OF REACH OF CHILDREN. IF SWALLOWED OR PLACED INSIDE ANY PART OF THE BODY SEEK IMMEDIATE MEDICAL ATTENTION.**
- INSTALL BATTERIES IN ACCORDANCE WITH ITS POLARITY SYMBOLS. DO NOT FIT BUTTON CELLS IN REVERSE.
- ENSURE THE BATTERY COMPARTMENT IS SECURED BY A SCREW OR SECURING DEVICE.
- IF THE BATTERY COMPARTMENT (DOES NOT CLOSE SECURELY, STOP USING THE PRODUCT AND KEEP IT AWAY FROM CHILDREN.
- DO NOT SHORT CIRCUIT THE BATTERY TERMINALS.
- REMOVE THE BATTERIES IF LEFT FOR LONG PERIODS.
- **KEEP BATTERIES AWAY FROM CHILDREN AND PETS.**
- DO NOT MIX USED AND NEW BUTTON CELLS
- **NEVER DISPOSE OF BUTTON CELLS OR BATTERIES IN A FIRE.**

BATTERY DISPOSAL

- SPENT OR EXPIRED BUTTON CELLS MUST BE DISPOSED SAFELY AWAY FROM CHILDREN ACCESS AND RECYCLED IN COMPLIANCE WITH YOUR LOCAL AUTHORITY REGULATIONS.
- FOLLOW THE BATTERY MANUFACTURER'S SAFETY, USAGE, AND DISPOSAL INSTRUCTIONS.

CARE AND CLEANING INSTRUCTIONS

- ALWAYS SWITCH OFF AND REMOVE BATTERIES BEFORE CLEANING.
- DO NOT IMMERSE IN WATER.
- DO NOT USE ABRASIVE CLEANERS OR SHARP UTENSILS TO CLEAN THIS APPLIANCE.
- WIPE SURFACES WITH A SOFT DRY CLOTH. DRY THOROUGHLY BEFORE FURTHER USE.
- STORE THE APPLIANCE IN A COOL, DRY PLACE.
- SHOULD WATER ENTER THE HEARING AMPLIFIER, REMOVE THE BATTERY AND PLACE THE HEARING AMPLIFIER IN A WARM PLACE TO DRY.
- IT IS VERY IMPORTANT TO INCREASE THE VOLUME CONTROL VERY GRADUALLY TO AVOID SUDDEN INCREASE IN SOUND.

DISPOSAL

- Coopers of Stortford use recyclable or recycled packaging where possible.
- Please dispose of all packaging, paper, cartons in accordance with your local authority recycling regulations.
- At the end of the product's lifespan please check with your local council authorised household waste recycling centre for disposal.

Plastics, polybags – this is made from the following recyclable plastic

Code & Symbol	 OTHER
Type of Plastic	Please Specify (includes ABS, Nylon)

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UK: The Enterprise Department Ltd.