Rechargeable Knee Massager

Item No: M857 v001: 16/05/24

Instructions for Use

Please read and retain these instructions for future reference

java
wellness

ENGLISH INSTRUCTIONS

Power Details:

Voltage details: 220-240V~

Frequency details: 50Hz-60Hz

Wattage details: 10W

Accessories:

1* Massager

1*USB charge cable

Please read this instruction manual thoroughly before starting and ensure you are familiar with the operation of your new Rechargeable Knee Massager

Caution
Hot surface

**CAUTION! THIS ELECTRICAL APPLIANCE'S EXTERNAL SURFACES
WILL GET HOT IN NORMAL USE.
ALLOW TO COOL BEFORE TOUCHING THE EXTERNAL SURFACES.**

Structure

Features

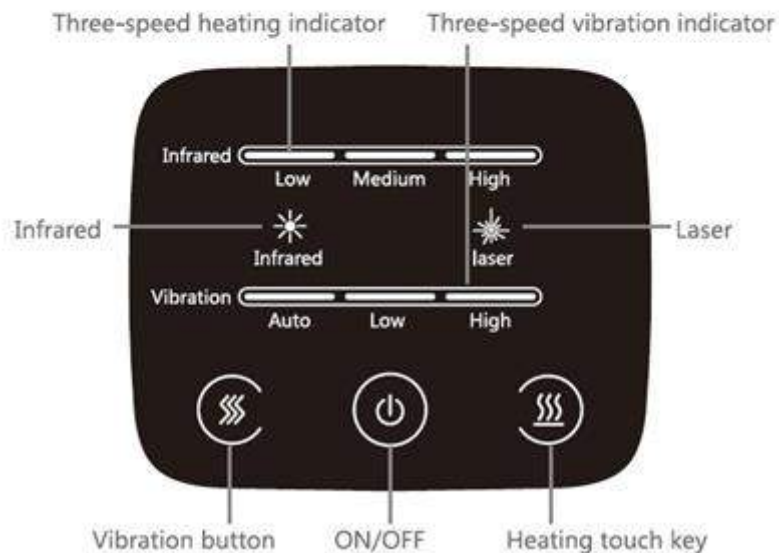
- | | | |
|---------------------------|---------------------|----------------------|
| ①: Indicator light | ④: Adjusting button | ⑦: Type-C power jack |
| ②: LED light touch screen | ⑤: Laser infrared | ⑧: Velcro |
| ③: Mesh cloth | ⑥: Vibration Area | |

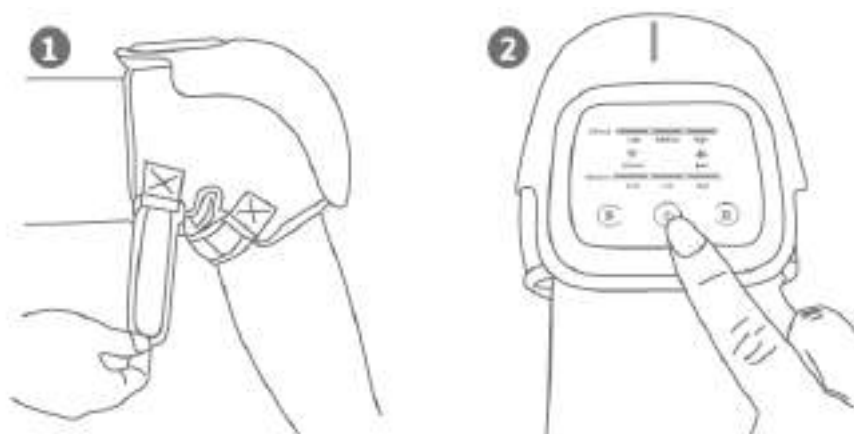
Parameter

- | | |
|----------------------------|-------------------------|
| •Rated voltage: 3.7V | •Rated Power: 10W |
| •Battery capacity: 3000mAh | •Input current: DC5V=2A |

Button panel introduction

- LED light touch screen, clearly visible and easy to operate
- Ergonomic design, fully wrapped to fit the knee joint
- Built-in 3 infrared laser lights, carbon fiber hot compress
- The buzzer prompts button is touched.
- 10 minutes timer.
- Low-grade heating 45 degrees, high-grade heating 55 degrees


1. Put the product on your knees, fasten the product on your knees with a fixing strap, and pull the product against your knees Near the fixed belt.
2. Long press "⏻" to turn on the Auto mode (vibration mode, low-grade heating, infrared , and laser) .
3. When powered on:
 Short press "⌘" to adjust the vibration mode (auto-low-high-turn off the vibration cycle in turn...), Lights turn white.
 Short press "⌘" to adjust the heating temperature (low-medium-high-close heating cycle in turn....), Lights turn white.
 Short press "⏻" to turn on and off the laser and infrared (turn on infrared-turn on the laser-turn off the infrared-turn off the laser cycle in turn...), Lights turn white.
4. Long press "⏻" to turn off the device.

Recharge

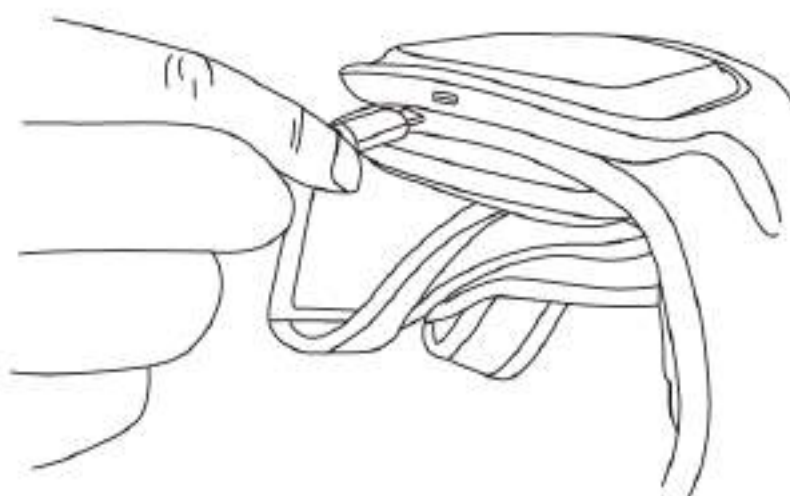1. Before the first use, please fully charge the product, and charging time about 3-4 hours
2. It is not recommended to completely drain the battery
3. When the battery light flashes, please charge in time

- Connect the Type-c cable to the power jack when charging

Precautions and maintenance –

Use failure	The reason	Solution
Product cannot normal work	The product has just completed a 10-minute timer massage and automatically switches to standby	Please press the power switch again to restart
	Too long use time causes the product to overheat and automatically cut off the power	Reuse the product after it has cooled
Product does not heat	When used outside the recommended ambient temperature range, the heating effect is not obvious	Recommended for use at an ambient temperature of 15-35 degrees Celsius
Product abnormal sound	This sound is caused by the mechanical structure of the product	This sound is normal and does not mean that the product function is malfunctioning
Stop during operation	The battery is exhausted	Connect the charging cable to charge
	Timed massage automatically stops after 10 minutes	Restart

Transport environment

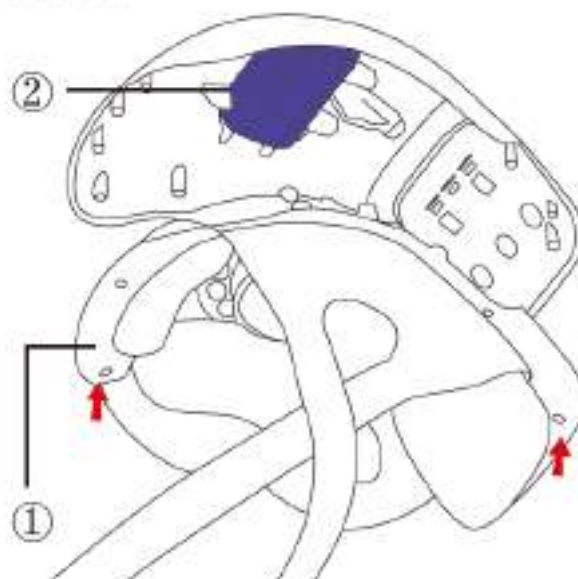
- Temperature: -20℃ ~ +50℃
- Relative humidity: ≤90%
- Atmospheric pressure: 70kPa ~ 106kPa

Working environment

- Temperature: +5℃ ~ +40℃
- Relative humidity: ≤80%
- Atmospheric pressure: 86kPa ~ 106kPa

Battery removal diagram

1. Unscrew the eleven screws at the bottom of the host
2. Open the cover structure and take out the battery fixed on the cover

WARNINGS AND INSTRUCTIONS:

• The main unit (including lithium battery) and accessories of this product have reached the end of their service life. Please do not discard them randomly, otherwise electronic waste will be generated to pollute the environment. Before discarding the product, the battery must be removed from the product; when removing the battery, the product must be disconnected Power; the battery should be disposed of safely when removed.

SAFETY

WHEN USING ELECTRICAL APPLIANCES, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED:

- READ ALL INSTRUCTIONS BEFORE USE.
- FOR DOMESTIC INDOOR USE ONLY.
- DO NOT USE OUTDOORS OR FOR COMMERCIAL USE.
- ONLY USE WITH THE MAINS ADAPTOR/CHARGER SUPPLIED.
- **DO NOT** USE ANY OTHER MAINS ADAPTOR/CHARGER AS THIS MAY CAUSE DAMAGE AND INVALIDATES ANY WARRANTY YOU MAY HAVE.
- BEFORE USE, CHECK THOROUGHLY FOR ANY DEFECTS AND DO NOT USE IF DEFECTS ARE FOUND. TAKE CARE NOT TO DROP THE APPLIANCE AS HEAVY IMPACTS MAY CAUSE INTERNAL DAMAGE.
- BEFORE CONNECTING THE APPLIANCE, CHECK IF THE VOLTAGE INDICATED ON THE BOTTOM OF THE APPLIANCE IS IN ACCORDANCE WITH THE MAINS VOLTAGE IN YOUR HOME.
- CHECK THE APPLIANCE AND POWER CABLE CAREFULLY FOR DAMAGE BEFORE EACH USE.
- THIS APPLIANCE CAN BE USED BY CHILDREN AGED FROM 8 YEARS AND ABOVE, AND PERSONS WITH REDUCED PHYSICAL, SENSORY OR MENTAL CAPABILITIES OR LACK OF EXPERIENCE AND KNOWLEDGE IF THEY HAVE BEEN GIVEN SUPERVISION OR INSTRUCTION CONCERNING USE OF THE APPLIANCE IN A SAFE WAY AND UNDERSTAND THE HAZARDS INVOLVED. CHILDREN SHALL NOT PLAY WITH THE APPLIANCE. CLEANING AND USER MAINTENANCE SHALL NOT BE MADE BY CHILDREN WITHOUT SUPERVISION.
- CHILDREN OF LESS THAN 3 YEARS SHOULD BE KEPT AWAY UNLESS CONTINUOUSLY SUPERVISED.
- **SOME PARTS OF THIS PRODUCT CAN BECOME VERY HOT AND CAUSE BURNS. PARTICULAR ATTENTION HAS TO BE GIVEN WHERE CHILDREN AND VULNERABLE PEOPLE ARE PRESENT.**
- THE USE OF ANY ACCESSORY OR ATTACHMENTS NOT RECOMMENDED BY COOPERS OF STORTFORD MAY CAUSE INJURIES AND INVALIDATE ANY WARRANTY YOU MAY HAVE.
- DO NOT LET THE POWER CABLE HANG OVER THE EDGE OF A TABLE OR COUNTER, OR TOUCH HOT SURFACES.
- DO NOT PLACE ON OR NEAR HEAT SOURCES.
- USE ON A LEVEL, STABLE SURFACE.
- DO NOT COVER THE ITEM IN ANY WAY WHEN IN USE AS THIS MAY CAUSE OVERHEATING.
- DO NOT USE THIS APPLIANCE FOR ANY OTHER USE THAN THE INTENDED USE IT IS DESIGNED FOR.
- ENSURE THERE IS SUFFICIENT DISTANCE BETWEEN THIS APPLIANCE AND OTHER ITEMS ON THE WORK SURFACE OR WORK STATION, OR SUFFICIENT DISTANCE BETWEEN THE APPLIANCE AND THE CONFINES OF THE AREA IN WHICH IT IS SITUATED. WE RECOMMEND A MINIMUM CLEARANCE AROUND ALL FACES OF AT LEAST 10CM.
- KEEP ITEM AT LEAST 1M AWAY FROM COMBUSTIBLE MATERIAL SUCH AS FURNITURE, CUSHIONS, BEDDING, PAPER, CLOTHES, CURTAINS ETC.
- TO PROTECT AGAINST ELECTRIC SHOCK, DO NOT IMMERSE POWER CABLE, MAINS ADAPTOR/ CHARGER OR APPLIANCE, IN WATER OR ANY OTHER LIQUID.
- UNPLUG FROM OUTLET WHEN NOT IN USE AND BEFORE CLEANING.
- DO NOT OPERATE ANY APPLIANCE WITH A DAMAGED POWER CABLE OR MAINS ADAPTOR/ CHARGER OR AFTER THE APPLIANCE MALFUNCTIONS, OR HAS BEEN DAMAGED IN ANY MANNER. WE RECOMMEND THAT THE POWER CABLE IS NOT KINKED OR STORED IN A TWISTED MANNER, AND THAT YOU REGULARLY CHECK ITS CONDITION.
- DO NOT LEAVE UNATTENDED.
- ALWAYS SWITCH OFF AFTER COOKING OR IF YOU HAVE TO LEAVE THE ROOM.
- THE USE OF OVEN GLOVES IS RECOMMENDED WHEN HANDLING HOT FOOD OR HOT APPLIANCES.
- ALLOW TO COOL BEFORE CARRYING OR PLACING IN STORAGE.
- ANY REPAIRS MUST BE CONDUCTED BY A QUALIFIED ELECTRICIAN OR ELECTRICAL SERVICE AGENT.

PLUG WIRING (UK & IRELAND)

This appliance is supplied with a mains adaptor/charger with BS1363 plug pins.

DO NOT use if the socket outlets in your home or office are not suitable for this product's mains adaptor/ charger plug pins. Attempts to insert the mains adaptor/charger into the wrong socket is likely to cause electric hazard or damage.

DO NOT use with any other mains transformer/charger than the one that is supplied with this product.

CARE AND CLEANING INSTRUCTIONS

- Always switch off and unplug before cleaning.
- Do NOT immerse in water. Risk of electric shock.
- Do NOT use abrasive cleaners or sharp utensils to clean this appliance.
- Wipe surfaces with a soft dry cloth.

- Allow to dry thoroughly before further use.
- Store the appliance in a cool, dry place.

BATTERIES

- Keep batteries away from children and pets.
- Install batteries in accordance with its polarity symbols.
- Never dispose of batteries in a fire.
- Battery disposal, spent or expired batteries must be properly disposed of and recycled in compliance with local regulations. For detailed information, contact your local authority.
- Follow the battery manufacturer's safety, usage, and disposal instructions.

DISPOSAL

- Coopers of Stortford use recyclable or recycled packaging where possible.
- Please dispose of all packaging, paper, cartons in accordance with your local authority recycling regulations.
- At the end of the product's lifespan please check with your local council authorised household waste recycling centre for disposal.
- Plastics, polybags – this is made from the following recyclable plastic.

Code & Symbol	
Type of Plastic	Low Density Polyethylene

- At the end of the product's lifespan please check with your local council authorised household waste recycling centre for disposal.

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The Enterprise Department Ltd.