# FOLDABLE EMS NECK MASSAGER

Item No: M858/ 62948

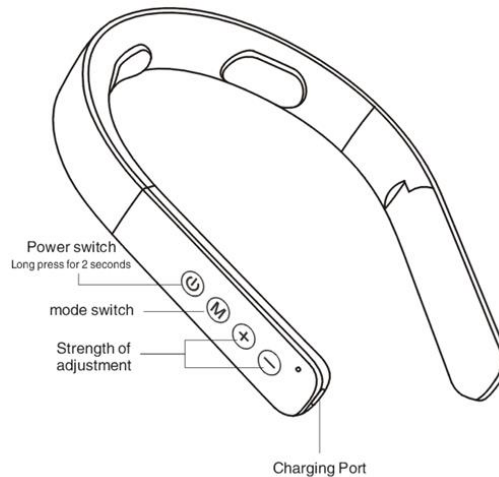
## Instructions for Use

v002: 08/10/24

Please read and retain these instructions for future reference

**JAVA**  
wellness

### ENGLISH INSTRUCTIONS

**WARNING! NOT TO BE USED BY PERSONS HAVING PACEMAKER.**

**Power Details:** Battery Type & details: LI-ion 500MAH  
Voltage details: 3.7V 5W  
Included / Not Included: INCLUDED

**ALWAYS USE ALKALINE BATTERIES.  
DO NOT USE LITHIUM BATTERIES.**

**Accessories:** USB Cable

Please read this instruction manual thoroughly before starting and ensure you are familiar with the operation of your new FOLDABLE EMS NECK MASSAGER.

#### Operation method

##### 1 Operation mode before USE:

Before use, first wet the skin of the cervical spine with a wet towel and then put the product around the neck, make the electrode close to the skin and adjust the appropriate massage intensity and mode. Then put the product around the neck, make the electrode close to the skin cervical spine, adjust position, set intensity and mode.

##### 2 Startup & Shutdown:

2.1 Longl"①"button on the product host press the" for two seconds, the host is turned on(the blue light is on when you hear a "di"sound),and enters the default mode, the default gear (1st gear).Indhc"© for two seconds and you will hear two beeps and the product will shut down.3 mode selection(currently 5 modes)

3.1 After the product host is turned on, short press mode key to switch modes. There are 15 modes inmode is incremented once, and the 15 modes can be switched cyclically(Note:M<sup>1</sup> mode,M2 modeM3 mode,M4 mode...M15 mode).Each time you switch a mode, the velocity returns to the default

1.4 gear(intensity)selection. Each time you press it, you will hear a beep, the strength will increase by one level and the strength will be divided into 20 levels (1,2...20).Press the"+"level again at the velocity of 20 to make two beeps, the velocity remains unchanged and the level of 20 is maintained.

4.2 The "-"on the main unit of the product is the force reduction button. Each time you press it you

hear a beep, the intensity decreases by one level and the force is divided into 20 levels (20,19,18...)

1),and the force is 1 Press the "-"file again at the same time to make two beeps, the intensity remains unchanged, and the level I is maintained.5 Heating options

5.1 Short press the "☺" button on the main unit of the product to turn on the heating button, press the switch key to turn on the heating function of the main unit with a beep, and turn on the heating function of the main unit with two beeps and it will gradually reach 42° within 5 seconds. Constant temperature. Short press the "☺"button on the main unit of the product again, and the heating function will stop with a beep.

## 6 Timing function

6.1 The product has an automatic 15 minute timer. The item will turn off after 15 minutes. It is recommended to use this item for no more than 30 minutes every day.

## 7 Low voltage alarm &charge

7.1 Low voltage alarm: When the product is short of power, the blue indicator light flashes quickly (Frequency 3 times per second),Insert one end of the charging cable into the product charging port, and the other end into the mobile phone charger

The red light flashes while charging, the red light stays on when it is full. The product cannot be used while charging.

7.2 The host battery: polymer lithium battery,capacity:450mah,with protection plate. current maintenance

\*When cleaning and maintaining this product please turn off the unit, clean the electrode with a wet paper towel and wipe and disinfect with a little medical alcohol (75%alcoholconcentration)when it is especially dirty.

★ When daily maintenance, please use neutral detergent and soft cloth to wipe, do not use corrosive detergent, do not put into the water to soak or rinse.

## Troubleshooting

\*If you have problems using the product, please refer to the following tips and solutions:

\*If the product is still in a fault condition or cannot be used, turn off the power.

★ Users should not disassemble or repair the product by themselves.

Dismantling or repairing by themselves will cause damage to the product and cause the invalidity of warranty. The company will not be responsible for any consequences caused thereby.

## FAQ

### Cause/Solution

Unable to boot 1.Low power 2.No power for remote control

Check the connection

### No feeling of massage

1. Ensure electrodes are touching the skin and skin is clean
2. Adjust the intensity

### Tingling

- 1.Use time is too long, recommended once 15 minutes, twice a day
- 2.Reduce massage intensity
- 3.skin allergy, stop use

### Weak

- 1.Adjust contact position
- 2.Cleanse skin and product
- 3.Increase the intensity

## Precautions

### Danger

**\*Please never use the product with the following medical electronic devices:**

- 1.Heart pacemakers and other implantable medical electronic devices;**
- 2.Artificial heart and lungs and other life-supporting medical electronic devices;**
- 3.Electrocardiographs and other medical electronic devices;**

**\*These medical electronic devices may cause malfunctions and can be life-threatening.**

**\*If the patient applies high-frequency surgical device and the product on the body at the same time, the electrode of the product may cause burns and damage to the product.**

**★ If the product is used near(1M)a short-wave or microwave therapy device, the output of the product may be unstable.\*The electrode should prevent contact with wounds and scars during treatment.**

**★ When you use the product, don't place the electrode near the chest heart, otherwise the risk of heart fibrillation will increase**

**When it is used by different user, clean the electrode on the product before using it.**

**★ When you use the product, stick the electrode tightly and evenly to the skin.\*The electrode of this product has a heat-generating function. When the hot compress time(more than 15 minutes),it may cause low temperature burns.**

**\*If you feel unwell during or after use, stop using it and seek medical attention.**

**\*In the following cases, consult your doctor before use:**

- 1.Patients under medical treatment and feel discomfort.**
- 2.Patients with abnormal heart and brain nerves**
- 3.Patients whose body temperature exceeds 38 degrees (fever period).**
- 4.Patients with infectious diseases.**
- 6.Patients under medication**
- 7.Patients with abnormal blood pressure.**
- 8.Patients who are not free to express their wishes.**
- 9.Patients with sensory impairment due too high peripheral circulatory disorders such as diabetes.**

**10.Patients with skin allergies to stainless steel electrodes.**

**★Usage under the following conditions is prohibited:**

- 1.Malignant tumor patients**
- 2.Patients with bleeding tendency**
- 4.Acute suppurative inflammation**
- 6.Patients with metal implant**
- 7.Near the heart**

**\*Don't apply it near the heart, above the neck, head, mouth, genitals, or skin disorders and don't treat the soles of the feet simultaneously In addition, don't compress the internal organs when using. Otherwise, an accident or physical discomfort may occur.**

**★ Don't use with other medical devices or more than 2 products Otherwise, nausea or physical discomfort may occur.\*Don't use for purposes other than the ones described in the instruction manual Otherwise, accident or malfunction may occur.\*Don't use the electrode with a chipped or damaged surface.**

**★ Don't use it for children or people who have no expressiveness, otherwise it may cause an accident or cause discomfort.\*Don't use it in a place with high humidity ,such as a bathroom, or use it while bathing Otherwise you may receive a strong electric shock and damaged the product.**

**\*Don't use it when you are sleeping.**

**\*Don't use while driving a car**

**\*Don't stick the electrode to the affected area for a long time otherwise it may cause skin inflammation.**

★ When massaging, don't use the metal parts of belts, necklaces, touch the electrode, otherwise you may receive a strong electric shock.

\*Don't apply the product on the same place for than 30 minutes as this may cause muscle fatigue at the massage site to cause physical discomfort.\*Please wear this product correctly and use it on the cervical spine. Wear it symmetrically Otherwise accidents or physical discomfort may occur.

\*When the product is working for a long time the surface will be hot which is normal.

### Suggestion

\*Don't use or store this product in the following locations:

Places subject to high temperature and humidity

Places with water

Dusty places

Places susceptible to vibration shocks

Strong electromagnetic places

### Packing List

Massager\*1

USB\*1

User Manual\*1

### WARNINGS AND SAFETY INSTRUCTIONS:

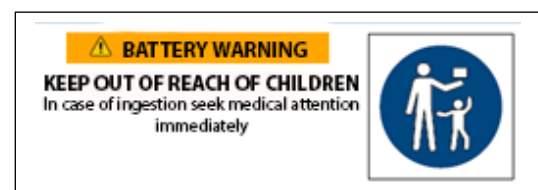
WHEN USING ANY APPLIANCE, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED:

- READ ALL INSTRUCTIONS BEFORE USE.
- FOR DOMESTIC INDOOR
- NOT FOR COMMERCIAL USE.
- BEFORE USE, CHECK THOROUGHLY FOR ANY DEFECTS AND DO NOT USE IF DEFECTS ARE FOUND. TAKE CARE NOT TO DROP THE APPLIANCE AS HEAVY IMPACTS MAY CAUSE INTERNAL DAMAGE.
- THE USE OF ANY ACCESSORY OR ATTACHMENTS NOT RECOMMENDED BY COOPERS OF STORTFORD MAY CAUSE INJURIES AND INVALIDATE ANY WARRANTY YOU MAY HAVE.
- DO NOT PLACE ON OR NEAR HEAT SOURCES.
- DO NOT COVER THE ITEM IN ANY WAY WHEN IN USE AS THIS MAY CAUSE OVERHEATING.
- DO NOT USE THIS APPLIANCE FOR ANY OTHER USE THAN THE INTENDED USE IT IS DESIGNED FOR.
- DO NOT LEAVE SWITCHED ON OR UNATTENDED.
- **WARNING! NOT TO BE USED BY PERSONS HAVING PACEMAKER.**

### BATTERIES

- ALWAYS USE ALKALINE BATTERIES. DO NOT USE LITHIUM BATTERIES.
- CONTAINS BUTTON CELLS

- **BATTERY WARNING: KEEP OUT OF REACH OF CHILDREN. IF SWALLOWED OR PLACED INSIDE ANY PART OF THE BODY SEEK IMMEDIATE MEDICAL ATTENTION.**
- INSTALL BATTERIES IN ACCORDANCE WITH ITS POLARITY SYMBOLS. DO NOT FIT BATTERIES IN REVERSE.
- DO NOT SHORT CIRCUIT THE BATTERY TERMINALS.
- REMOVE THE BATTERIES IF LEFT FOR LONG PERIODS.
- **KEEP BATTERIES AWAY FROM CHILDREN AND PETS.**
- DO NOT MIX USED AND NEW BATTERIES.

- **NEVER DISPOSE OF BATTERIES IN A FIRE.**
- BATTERY DISPOSAL, SPENT OR EXPIRED BATTERIES MUST BE DISPOSED OF AND RECYCLED IN COMPLIANCE WITH YOUR LOCAL AUTHORITY REGULATIONS.
- FOLLOW THE BATTERY MANUFACTURER’S SAFETY, USAGE, AND DISPOSAL INSTRUCTIONS.

**CARE AND CLEANING INSTRUCTIONS**

- ALWAYS SWITCH OFF AND REMOVE BATTERIES BEFORE CLEANING.
- DO NOT IMMERSE IN WATER.
- DO NOT USE ABRASIVE CLEANERS OR SHARP UTENSILS TO CLEAN THIS APPLIANCE.
- WIPE SURFACES WITH A SOFT DRY CLOTH. DRY THOROUGHLY BEFORE FURTHER USE.
- STORE THE APPLIANCE IN A COOL, DRY PLACE.

**BATTERY DISPOSAL**

- SPENT OR EXPIRED BUTTON CELLS MUST BE DISPOSED SAFELY AWAY FROM CHILDREN ACCESS AND RECYCLED IN COMPLIANCE WITH YOUR LOCAL AUTHORITY REGULATIONS.
- FOLLOW THE BATTERY MANUFACTURER’S SAFETY, USAGE, AND DISPOSAL INSTRUCTIONS.

**DISPOSAL**

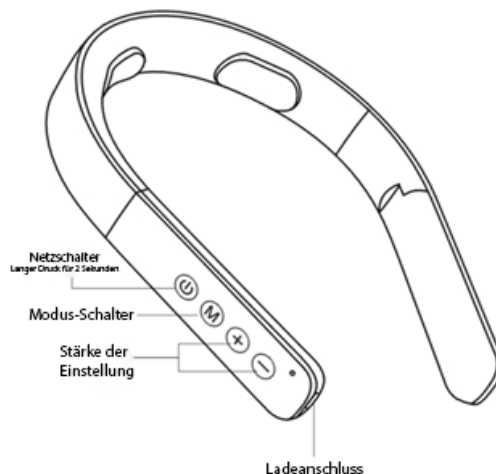
- Coopers of Stortford use recyclable or recycled packaging where possible.
- Please dispose of all packaging, paper, cartons in accordance with your local authority recycling regulations.
- At the end of the product’s lifespan please check with your local council authorised household waste recycling centre for disposal.
- Plastics, polybags – this is made from the following recyclable plastic.

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Code & Symbol	 OTHER
Type of Plastic	Please Specify (includes ABS, Nylon)

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UK: The Enterprise Department Ltd.