Set of 2 Suction Ear Cleaners

Item No: P125

Instructions for Use

v001: 19/08/24

Please read and retain these instructions for future reference

Java
wellness

ENGLISH INSTRUCTIONS

INSTRUCTIONS FOR USE:

- a. Open the package and take out the product.
- b. Prepare a bowl of lukewarm water.
- c. Insert the syringes into the water and squeeze the bulb hard to commence suction.
- d. Once the bulb is full, tilt the head slightly so the ear can be irrigated.
- e. Gently insert the tip of the syringe into the ear, ensuring it is not directed at the eardrum.
- f. Very gently, squeeze the bulb of the ear syringe to flush/irrigate the ear.
- g. Use a soft towel to dry the area around the ear.
- h. Individual use only.
- i. Before & After use: Clean syringe with warm soapy water and rinse thoroughly after each use. Shake out excess water and let air-dry thoroughly.

Disinfecting Type: Use Alcohol to wipe the surface of object, Alcohol wipes (70-90%) after each use. Ensure that no liquid enters the interior of the device.

[Caution]

- Please read through these instructions carefully in order for you to understand all functions and safety information.
- Ear syringe is a device intended for healthcare purposes by non-professional users. Pay attention to the application scope and application method when using the product.
- For individual use, do not share with others.
- After use, please incinerate or dispose the product according to relevant national regulations.
- Please check the validity of the product before use.

[Warning]

- If you are experiencing discomfort after use, discontinue use and contact your healthcare professional. The device is not replacing the need for the consultation of a physician, especially if not matching the patients' symptoms. Calling a doctor or an ambulance is advised if needed.
- Do not use boiling water or strong chemicals in the bulb.
- Do not place the device on radiator or contact with hot metal.
- Do not expose the device to direct sunlight or heat.

CARE AND CLEANING INSTRUCTIONS:

- Do NOT use abrasive cleaners or sharp utensils to clean this product.
- Store the item in a cool, dry place.
- Wipe with a clean dry cloth.

DISPOSAL

- Coopers of Stortford use recyclable or recycled packaging where possible.
- Please dispose of all packaging, paper, cartons in accordance with your local authority recycling regulations.
- At the end of the product’s lifespan please check with your local council authorised household waste recycling centre for disposal.
- Plastics, polybags – this is made from the following recyclable plastic.

Code & Symbol	 PVC
Type of Plastic	Polyvinyl Chloride

- At the end of the product’s lifespan please check with your local council authorised household waste recycling centre for disposal.

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**The Enterprise Department Ltd.