Slimline Knee & Shoulder Heated Massager

Item No: P129 / xxxxxx

Instructions for Use

v001: 01/08/24

Please read and retain these instructions for future reference

java

wellness

ENGLISH INSTRUCTIONS

Power Details: 5V DC 1A

Wattage details: 7.5W

Accessories: 1xusb cable,1xmanual

Please read this instruction manual thoroughly before starting and ensure you are familiar with the operation of your new Slimline Knee & Shoulder Heated Massager

User Instructions

Work Environment

	Work Environment	Storage and transportation
Temperature range humidity	0°C~40°C	0°C~50°C
Range atmospheric pressure	< 80%RH 70KPA -106KPA	< 80%RH 70KPA -106KPA

Multipurpose


- Excessive massage is harmful to the body.
- This product will automatically shut down after 15 minutes of use. Please adjust the usage time according to your actual situation.
- When starting to use this product, it is recommended to choose low-temperature mode.
- Mild skin redness, swelling, and fever after use are normal phenomena. Please feel free to use. However, if severe skin redness and swelling occur after use, please stop using immediately and consult a doctor.
- It is recommended to avoid direct contact with the skin. The temperature of the product is very high during long-term use. Be careful of burns. The heating function cannot be used continuously for more than 15 minutes to avoid burns. It is recommended to start at the lowest temperature and gradually increase it according to personal conditions, with an appropriate temperature to avoid skin burns for people who are not sensitive to heat.
- If the user experiences any discomfort during use, such as dizziness, headache, asthma, etc. Please immediately stop using and analyze the cause of discomfort.
- When not in use, turn off the machine and keep it properly.
- This product is a household appliance and cannot replace medical equipment or drugs.

Please note that incorrect operation may lead to safety accidents or property damage.

Warning:

Please consult a doctor before use:

- Patients with osteoporosis, fractures, sprains, acute pain, and other leg diseases;
- Patients with leg nerve disorders or skin sensory disorders; Pregnant or postpartum women;
- Heart disease patients;
- Massage has a heating surface, so people who are not sensitive to heat must pay attention.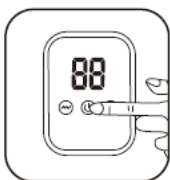

1. Long press the switch to turn on the machine.


2. Secure the machine in the knee/elbow/shoulder position.


3.

- Press the temperature button to turn on the heating function. Press 1 gain to adjust the temperature, and the corresponding heating indicator light will change accordingly
- The first white indicator light indicates low temperature; The second white indicator light is on, indicating the medium temperature;
- The third white indicator light comes on, indicating high temperature. The absence of a white indicator light indicates that the heating function is turned off.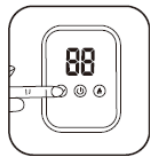

4.

- Press the vibration button to turn on the vibration function, press again to display only the vibration intensity, and the corresponding vibration indicator will change accordingly
- The first white indicator light 1 is on, indicating low vibration intensity; The second white indicator light comes on, indicating moderate vibration intensity:
- The third white indicator light comes on, indicating high vibration intensity. The absence of a white indicator light indicates that the vibration function is turned off.

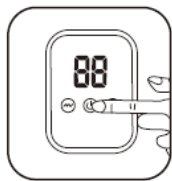5.Long press the power button to shut down.

WARNINGS AND SAFETY INSTRUCTIONS:

WHEN USING ELECTRICAL APPLIANCES, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED:


- READ ALL INSTRUCTIONS BEFORE USE.
- FOR DOMESTIC INDOOR USE ONLY.
- DO NOT USE OUTDOORS OR FOR COMMERCIAL USE.
- ONLY USE WITH THE MAINS ADAPTOR/CHARGER SUPPLIED.
- **DO NOT** USE ANY OTHER MAINS ADAPTOR/CHARGER AS THIS MAY CAUSE DAMAGE AND INVALIDATES ANY WARRANTY YOU MAY HAVE.
- BEFORE USE, CHECK THOROUGHLY FOR ANY DEFECTS AND DO NOT USE IF DEFECTS ARE FOUND. TAKE CARE NOT TO DROP THE APPLIANCE AS HEAVY IMPACTS MAY CAUSE INTERNAL DAMAGE.
- BEFORE CONNECTING THE APPLIANCE, CHECK IF THE VOLTAGE INDICATED ON THE BOTTOM OF THE APPLIANCE IS IN ACCORDANCE WITH THE MAINS VOLTAGE IN YOUR HOME.
- CHECK THE APPLIANCE AND POWER CABLE CAREFULLY FOR DAMAGE BEFORE EACH USE.
- THIS APPLIANCE CAN BE USED BY CHILDREN AGED FROM 8 YEARS AND ABOVE, AND PERSONS WITH REDUCED PHYSICAL, SENSORY OR MENTAL CAPABILITIES OR LACK OF EXPERIENCE AND KNOWLEDGE IF THEY HAVE BEEN GIVEN SUPERVISION OR INSTRUCTION CONCERNING USE OF THE APPLIANCE IN A SAFE WAY AND UNDERSTAND THE HAZARDS INVOLVED. CHILDREN SHALL NOT PLAY WITH THE APPLIANCE. CLEANING AND USER MAINTENANCE SHALL NOT BE MADE BY CHILDREN WITHOUT SUPERVISION.
- CHILDREN OF LESS THAN 3 YEARS SHOULD BE KEPT AWAY UNLESS CONTINUOUSLY SUPERVISED.
- DO NOT PLACE ON OR NEAR HEAT SOURCES.
- DO NOT COVER THE ITEM IN ANY WAY WHEN IN USE AS THIS MAY CAUSE OVERHEATING.
- DO NOT USE THIS APPLIANCE FOR ANY OTHER USE THAN THE INTENDED USE IT IS DESIGNED FOR.
- TO PROTECT AGAINST ELECTRIC SHOCK, DO NOT IMMERSE POWER CABLE, MAINS ADAPTOR/ CHARGER OR APPLIANCE, IN WATER OR ANY OTHER LIQUID.
- DO NOT OPERATE ANY APPLIANCE WITH A DAMAGED POWER CABLE OR MAINS ADAPTOR/ CHARGER OR AFTER THE APPLIANCE MALFUNCTIONS OR HAS BEEN DAMAGED IN ANY MANNER. WE RECOMMEND THAT THE POWER CABLE IS NOT KINKED OR STORED IN A TWISTED MANNER, AND THAT YOU REGULARLY CHECK ITS CONDITION.
- DO NOT LEAVE UNATTENDED.
- ANY REPAIRS MUST BE CONDUCTED BY A QUALIFIED ELECTRICIAN OR ELECTRICAL SERVICE AGENT.

CARE AND CLEANING INSTRUCTIONS

- Always switch off and unplug before cleaning.
- Do NOT immerse in water. Risk of electric shock.
- Do NOT use abrasive cleaners or sharp utensils to clean this appliance.
- Wipe surfaces with a soft dry cloth.
- Allow to dry thoroughly before further use.
- Store the appliance in a cool, dry place.

DISPOSAL

- Coopers of Stortford use recyclable or recycled packaging where possible.
- Please dispose of all packaging, paper, cartons in accordance with your local authority recycling regulations.
- At the end of the product's lifespan please check with your local council authorised household waste recycling centre for disposal.
- Plastics, polybags – this is made from the following recyclable plastic.

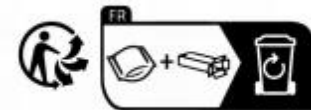Code & Symbol	 OTHER
Type of Plastic	Please Specify (includes ABS, Nylon)

- At the end of the product's lifespan please check with your local council authorised household waste recycling centre for disposal.

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The Enterprise Department Ltd.