MINI ELECTRIC EMS MASSAGER

Item No: P262/60626

Instructions For Use

v001: 18/09/24

Please read and retain these instructions for future reference

INSTRUCTIONS FOR USE:

Step by Step Instructions & Photos

Voltage	3.7V
How to use	A-F (8 modes) 19 gears
Frequency	1~100HZ
Working hours	15 minutes
Output	35mA maximum
Size	62x62x16
Weight	31
Battery capacity	150mAH

Power Details:


Voltage details: 3.7V

Frequency details: 1Hz-100Hz

Accessories:

<p>EMS Mini Massage 1PC</p> 	<p>Massage Sticker 3PCS</p> 	<p>Charging cable 1PC</p> 
---	---	---

Please read this instruction manual thoroughly before starting and ensure you are familiar with the operation of your new MINI ELECTRIC EMS MASSAGER.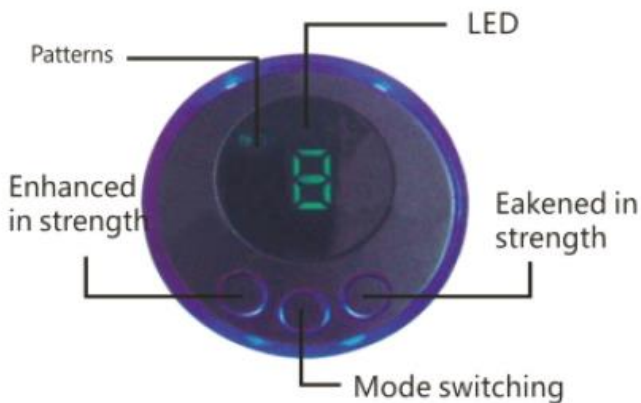

 **Dangerous**

Do not use this product with the following medical electronic instruments. Otherwise, it may cause accidents or malfunction of medical electronic instruments, and bring serious damage to the body.


Note: Take off your socks when using them, put both barefoot on the mat at the same time, and then press the Start button.

 **Dangerous**

Do not use the following personnel. Otherwise, it may cause accidents or skin and body problems.

Pregnant women, women who have just given birth, and all electronic rejection products on patients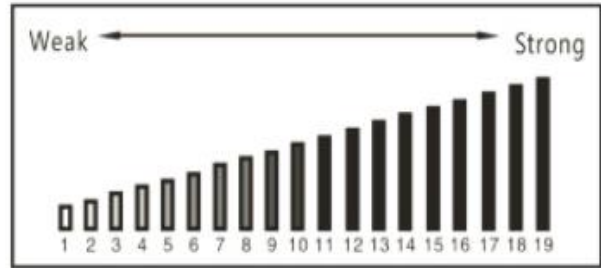

Technical parameter

(GB)MINI ELECTRIC EMS MASSAGER

LCD Model (8 modes , 19 levels)

1.8 Modes :

When the machine is switched to level 1-19, it will automatically shut down once every 15 minutes.

2.19 Levels Adjustment :

If it's switched to level 1-19 when the unit is in use, it will automatically switch to level 0 in 5 seconds , and switch off automatically after level 0 in 20 seconds. The right light is one when charging (about 1-2 hours). The red light will be off when it's full.

3.6 Mode

Please set a lower intensity when using it for the first time, and adjust after adapting to the low intensity. These are 6 modes in total.

(GB)How to use:

1. It can be used for about 160 minutes after one hour of charging. The greater the intensity, the greater the power consumption will be. Please note that you cannot use it during charging and can only be used after charging.
2. DO NOT leave your feet otherwise it will automatically shut down. Put your feet up and press the power button. Please be careful not to wear socks and remove the plastic bag. DO NOT leave with one foot otherwise it will automatically shut down. Remember to unplug it after charging, otherwise it won't be able to boot.
3. The button of the main unit is buckled on the mat like a button. Take off your socks and place your feet on the footprint. Press the "+" button to turn on the machine and press twice/intensify "-" button indicates reduced intensity /shutdown indicates switching mode.
4. The stimulation intensity is divided into 19 levels. (Intensity from 0 to 19 levels).
5. When using it for the first time, please set a lower strength to start then after adaption to the low strength, please debug other strengths.
6. As the remaining battery capacity gradually decreases, the current stimulation will also gradually weaken. If you continues to use it, please charge the main unit.

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Debugging

Conditions	Possible reasons	Solutions
Fail to connect power	The main unit battery is exhausted	Please charge the host computer
	Used in low or high temperature environment	It is recommended to use the ambient temperature at (15~35°C)
No electrical stimulation	The power is not connected	Short press the on or off button until the LED1 light is on
Current stimulation is too weak	Setting the intensity is low	Press harder to improve intensity
Current stimulation is too strong	Setting the intensity is too high	The compression is weakened and the intensity is reduced
Current stimulation is suddenly interrupted	The remaining power of the host battery is exhausted	Please charge the host computer

DISPOSAL

- Coopers of Stortford use recyclable or recycled packaging where possible.
- Please dispose of all packaging, paper, cartons in accordance with your local authority recycling regulations.
- At the end of the product’s lifespan please check with your local council authorised household waste recycling centre for disposal.
- Plastics, polybags – this is made from the following recyclable plastic.

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The Enterprise Department Ltd.
11 Bridge Street, Bishops Stortford
Herts, UK, CM23 2JU
Tel: 0330 331 0300

NI/EU:
D.S.B. LD, 160 Bd. de Fourmies, 59100 Roubaix, France
Tel: (+44) 1279 701269

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