OUTER EAR RECHARGABLE HEARING AMPLIFIER

Item No: P269 / 67479

Instructions for Use

v001: 26/09/24

Please read and retain these instructions for future reference

ENGLISH INSTRUCTIONS

Power Details:

Battery Type & details: 1* rechargeable battery 40 mAh

Voltage details: 1.2V

Included

Cleaning Brush

FIGURE A

Adapter

FIGURE B

Data Line

FIGURE C

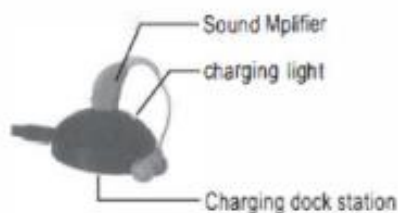
FIGURE D

Six(6) Silicone Adjustable Tips

FIGURE E

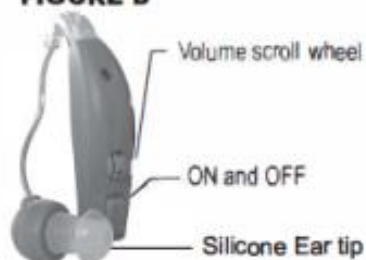
FIGURE F

Accessories: Charging base *1, Brush*1, Earplugs*6, USB*1, Plug*1

Please read this instruction manual thoroughly before starting and ensure you are familiar with the operation of your new OUTER EAR RECHARGABLE HEARING AMPLIFIER.

GETTING STARTED

It is important to follow the instructions (1-3) below for the first two times charging the Rechargeable Sound Amplifier

1. Slide the switch down located on the back of the earpiece to turn the Rechargeable Sound Amplifier OFF.
2. Connect the adapter's cord into the rechargeable docking station plug the opposite end of the adapter's cord into an outlet. Upon plugging, the red light on the adapter (Figure C) will illuminate and the green light on the docking station (Figure D) will light simultaneously.
3. Place the Rechargeable Sound Amplifier into the recharging docking station. The green light on the docking station will turn red if connected properly.

Note: DO NOT remove the Rechargeable Sound Amplifier from the docking station until the light turns green (again)- signalling fully charged.

Important: Follow the steps below to ensure longer time.

DIRECTIONS

How to wear the Rechargeable Sound Amplifier

1. The Rechargeable Sound Amplifier comes with an additional 6 different silicone ear tips.
2. Select the silicone tip that best fits your ear and place onto the Rechargeable Sound Amplifier
3. Rechargeable Sound Amplifier can be worn in either the right or left ear.
4. Below the volume scroll wheel is the on/off switch. This must be in the "on" position or in the "up" position for use and slide down "OFF" turn the device OFF.
5. Take the earpiece and place it over the top of the desired ear. The earpiece will curve around and fit behind the ear.
6. Then take the ear bud of the Rechargeable Sound Amplifier Sound Amplifier which is at the end of the small wire and carefully insert it into the ear canal.
7. Adjust the volume to the desired level.


WARNINGS AND SAFETY INSTRUCTIONS:

WHEN USING ANY APPLIANCE, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED:

- READ ALL INSTRUCTIONS BEFORE USE.
- NOT FOR COMMERCIAL USE.
- BEFORE USE, CHECK THOROUGHLY FOR ANY DEFECTS AND DO NOT USE IF DEFECTS ARE FOUND. TAKE CARE NOT TO DROP THE APPLIANCE AS HEAVY IMPACTS MAY CAUSE INTERNAL DAMAGE.
- THE USE OF ANY ACCESSORY OR ATTACHMENTS NOT RECOMMENDED BY COOPERS OF STORTFORD MAY CAUSE INJURIES AND INVALIDATE ANY WARRANTY YOU MAY HAVE.
- DO NOT PLACE ON OR NEAR HEAT SOURCES.
- DO NOT COVER THE ITEM IN ANY WAY WHEN IN USE AS THIS MAY CAUSE OVERHEATING.
- DO NOT USE THIS APPLIANCE FOR ANY OTHER USE THAN THE INTENDED USE IT IS DESIGNED FOR.
- DO NOT LEAVE SWITCHED ON OR UNATTENDED.

BATTERIES

- **ALWAYS USE ALKALINE BATTERIES. DO NOT USE LITHIUM BATTERIES.**
- **CONTAINS BUTTON CELLS**

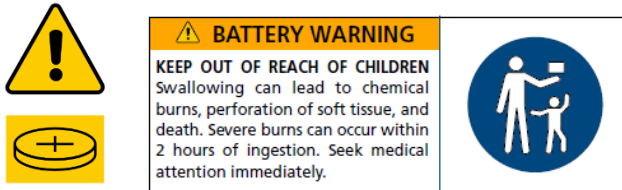- **BATTERY WARNING: KEEP OUT OF REACH OF CHILDREN. IF SWALLOWED OR PLACED INSIDE ANY PART OF THE BODY SEEK IMMEDIATE MEDICAL ATTENTION.**
- INSTALL BATTERIES IN ACCORDANCE WITH ITS POLARITY SYMBOLS. DO NOT FIT BATTERIES IN REVERSE.
- DO NOT SHORT CIRCUIT THE BATTERY TERMINALS.
- REMOVE THE BATTERIES IF LEFT FOR LONG PERIODS.
- **KEEP BATTERIES AWAY FROM CHILDREN AND PETS.**
- DO NOT MIX USED AND NEW BATTERIES.
- **NEVER DISPOSE OF BATTERIES IN A FIRE.**
- BATTERY DISPOSAL, SPENT OR EXPIRED BATTERIES MUST BE DISPOSED OF AND RECYCLED IN COMPLIANCE WITH YOUR LOCAL AUTHORITY REGULATIONS.
- FOLLOW THE BATTERY MANUFACTURER'S SAFETY, USAGE, AND DISPOSAL INSTRUCTIONS.

CARE AND CLEANING INSTRUCTIONS

- ALWAYS SWITCH OFF AND REMOVE BATTERIES BEFORE CLEANING.
- DO NOT IMMERSE IN WATER.
- DO NOT USE ABRASIVE CLEANERS OR SHARP UTENSILS TO CLEAN THIS APPLIANCE.
- WIPE SURFACES WITH A SOFT DRY CLOTH. DRY THOROUGHLY BEFORE FURTHER USE.
- STORE THE APPLIANCE IN A COOL, DRY PLACE.

BATTERY DISPOSAL

- SPENT OR EXPIRED BUTTON CELLS MUST BE DISPOSED SAFELY AWAY FROM CHILDREN ACCESS AND RECYCLED IN COMPLIANCE WITH YOUR LOCAL AUTHORITY REGULATIONS AND NOT INTO GENERAL WASTE.
- FOLLOW THE BATTERY MANUFACTURER'S SAFETY, USAGE, AND DISPOSAL INSTRUCTIONS.

DISPOSAL

- Coopers of Stortford use recyclable or recycled packaging where possible.
- Please dispose of all packaging, paper, cartons in accordance with your local authority recycling regulations.
- At the end of the product's lifespan please check with your local council authorised household waste recycling centre for disposal.
- Plastics, polybags – this is made from the following recyclable plastic.

Code & Symbol	 OTHER
Type of Plastic	Please Specify (includes ABS, Nylon)

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The Enterprise Department Ltd.