# Posture Pro Corrector

Item No: P989 / 76910

## Instructions for Use


v001: 25/09/25

Please read and retain these instructions for future reference

# Java

wellness

### ENGLISH INSTRUCTIONS

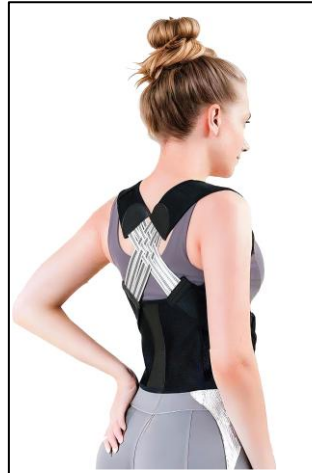### INSTRUCTIONS FOR USE:

Step by Step Instructions:

1. Place the posture corrector on your shoulders by putting your arms through the two loops.
2. Make sure the central section is positioned in the middle of the back. The cross section is to be at the back.
3. There are two touch fasten straps (one on each side) which will need to be pulled forward and placed on the support belt around your waist. Adjust them to a suitable comfort level where you can feel the support and shoulders are pulled back

Start by wearing the posture corrector 15-30 minutes a day. Gradually increase the duration until you reach 1-2 hours a day.

Do not wear corrector more than 2 hours a day to avoid muscle pain.

#### CARE AND CLEANING INSTRUCTIONS:

- Do NOT use abrasive cleaners or sharp utensils to clean this product.
- Store the item in a cool, dry place.
- Wipe with a clean dry cloth.

#### DISPOSAL

We use recyclable or recycled packaging where possible

Please dispose of all packaging in accordance with your local authority's recycling regulations

At the end of the product's lifespan check with your local council authorized household waste recycling centre for disposal.

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The Enterprise Department Ltd, Trading as Coopers of Stortford.