

## SEATED WALKING STICK

Item No: R173

## Instructions for Use

v001: 01/12/2025

Please read and retain these instructions for future reference

# Java

wellness



### INSTRUCTIONS FOR USE:



#### Height Adjustment

- On each leg, press the adjustment button in and pull or push the bottom of each leg to get to the desired height.
- There are 5 height settings.
- Ensure the silver button is locked into the hole position and that all three are placed at equal levels.

#### Stool Folding & Deployment

When the stool is folded, unsnap the small fastener on the underside of the seat, pull the bottom of the seat outwards to allow the three legs to expand to the seated position. Sit on it only after it is fully locked in place.

#### Folding

Lift the seat from the back, where it meets the cane, then fold it inward and snap the fastener back into place to secure the stool to the cane.

#### Weight Capacity

- Stool: 300 lbs (approx. 136.1 kg)
- Handle: 330 lbs (approx. 149.7 kg)

#### Precautions

- Align the button holes when adjusting the height.
- Use the stool only after it is fully locked.
- Avoid using on slippery or sloped surfaces.

**CARE AND CLEANING INSTRUCTIONS:**

- Do NOT use abrasive cleaners or sharp utensils to clean this product.
- Store the item in a cool, dry place.
- Wipe with a clean dry cloth.


**DISPOSAL**

We use recyclable or recycled packaging where possible

Please dispose of all packaging in accordance with your local authority’s recycling regulations

At the end of the product’s lifespan check with your local council authorized household waste recycling centre for disposal.

Plastic Polybags – this is made from the following recyclable plastic


Code & Symbol	 <b>OTHER</b>
Type of Plastic	Please Specify (includes ABS, Nylon)

**Distributed by / Verteilt durch / Distribué par/ Distribútor / Distribuuje.**

The Enterprise Department Ltd, Trading as Coopers of Stortford.

**UK address:** 11 Bridge Street, Bishops Stortford, Herts, UK, CM23 2JU

**EU address:** D.S.B. LD, 160 Bd. De Fourmies, 59100 Roubaix, France

 0330 331 0300

 [enquiries@coopersofstortford.co.uk](mailto:enquiries@coopersofstortford.co.uk)


 <https://www.coopersofstortford.co.uk>

**3Pagen**

Hergestellt für 3Pagen

Sankt-Joeris-Strasse 16-28

52477 Alsdorf

 +49 (0) 2404/980-204

 [kontakt@3pagen.de](mailto:kontakt@3pagen.de)

 [www.3pagen.de](http://www.3pagen.de)



Points de collecte sur [www.quefairedemesdechets.fr](http://www.quefairedemesdechets.fr)  
Privilégiez la réparation ou le don de votre appareil !