

WARNING

Always check the Safety Handle before use to make sure it is firmly secure. Any suction cup item will lose adhesion over time, so if there is any movement, re fix the Safety Handle before use. It is advisable to manually test the Handle before each use to ensure that it has not loosened over time.

THE SAFETY HANDLE IS TO BE USED TO ASSIST YOUR BALANCE ONLY; IT IS NOT DESIGNED TO TAKE THE WEIGHT OF A PERSON.


- Only fix to smooth, dry and secure surfaces.
- Do not allow the cups to bridge gaps or cracks in tiles etc. the cups must be placed over a completely smooth, non porous surface.
- Locate the best site for use, ensure the site and pads are clean of all dust, dirt and importantly soap build up.
- With the latches open, position the Safety Handle. Apply firm pressure to the Handle against the surface, expelling any excess air, and then flip the latches down securing the handle in place, if the Handle is loose repeat the process.
- Reverse the process to remove, if the cups stick to a surface; gently prise them off with a thin blunt instrument to break the seal.

DISPOSAL

- Coopers of Stortford use recyclable or recycled packaging where possible.
- Please dispose of all packaging, paper, cartons, packaging in accordance with your local recycling regulations.
- At the end of the product's lifespan please check with your local council authorised household waste recycling centre for disposal.



- Plastics, polybags – Contains the following recyclable plastic.

Code & Symbol	 OTHER
Type of Plastic	ABS
Commonly used for	CD's, crisp packets, various flexible packaging, baby bottles, sunglass lenses
Notes	Not often or widely recycled