


# Digital in Ear Hearing Amplifier Instructions for Use

7638

Please read and retain these instructions for future reference

v004: 25/05/22



## List of Contents

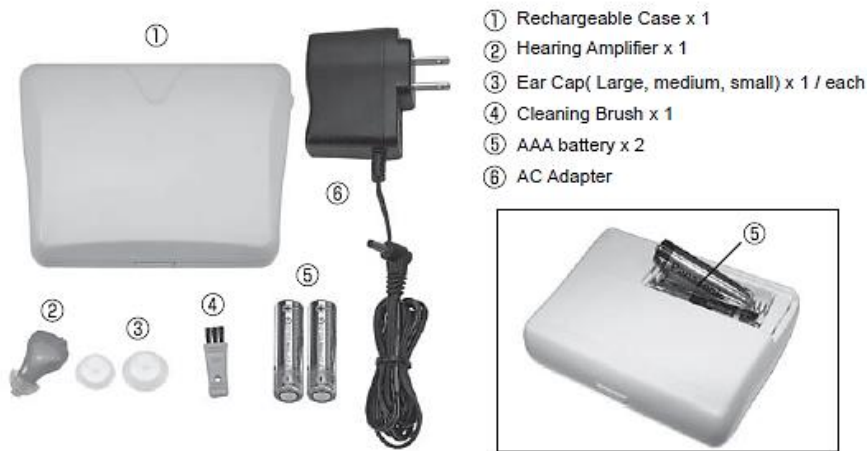

## Hearing Amplifier:

**Hollow of sound :**  
amplified sound comes from the hollow.

**Dial of volume adjustment :**  
you can turn the dial to adjust the volume.  
The power will be off when turned to the lowest volume.

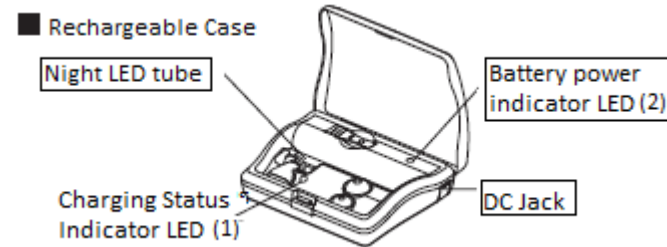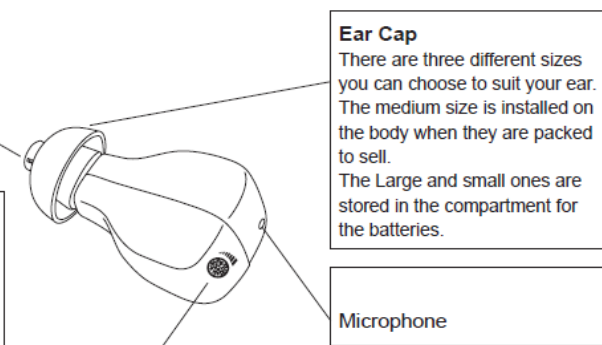

LEDs will light up when opening rechargeable case.

Both LEDs (1 & 2) green  
LED 2 green

LED 2 red

No light

Charging

Either Charging complete

Or no contact with amplifier, re-position  
Battery needs replacing.

Use AC adapter instead or replace  
batteries.

No batteries in compartment

Use adapter or install 2x AAA batteries

## Instructions

Read instructions carefully to ensure proper use and care of your rechargeable hearing amplifier:


**To Charge: Place hearing amplifier in the charging case as shown.**



- ① Insert hearing amplifier vertically as shown.
- ② Lay hearing amplifier down as shown by gently pushing towards the charging case cover.
- ③ Close charging case lid.

**Note:** Ensure that 2AAA batteries are fitted correctly. Please note the polarities when connecting batteries.

**Charge Status: Open the rechargeable case to see the indicator lamps**

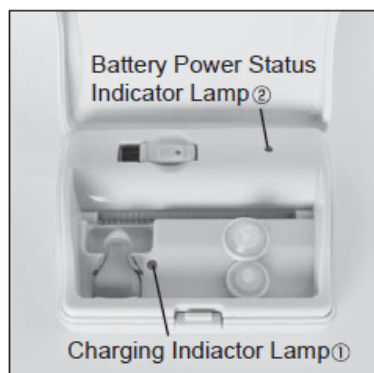

Green light on both Lamp ① and Lamp ②  
Indicates proper recharging conditions.  
(It can't be charged while the power is off.)



Lamp ① will turn off when fully charged.  
Green light on lamp ② only:

Indicates the charge is complete.

Lamp ② turn green light into red light.

Indicates the charging case batteries are low.  
Please replace the battery or use the AC  
adapter according to the enclosed instructions.

**Note: Before and after each use.**

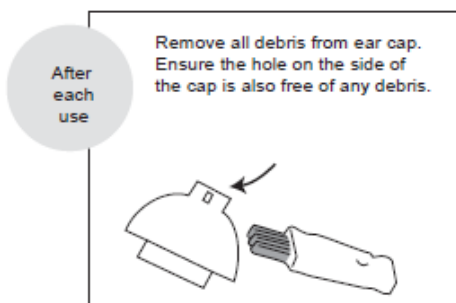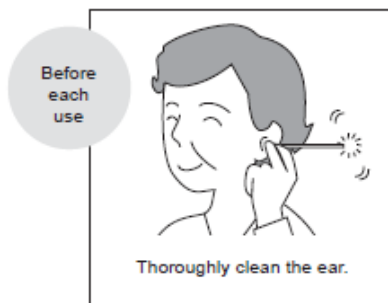






**Please read and follow all instructions carefully. Your hearing amplifier is a very delicate instrument.**

**Debris on the ear cap may cause improper function.**

**Repair and replacement parts: Please contact Customer Service to request repair service or purchase consumable parts.**

**Components Parts List**

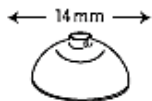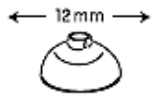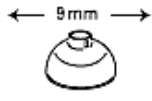Component	Item Number	Image
Hearing Amplifier	10.5 x 14.4 x 24 mm	 Shown with ear cap
Charging Case	96x 73 x30 mm	
Ear Cap(Large)	55 x 50 x17 mm	
Ear Cap(Medium)	55 x 50 x17 mm	
Ear Cap(Small)	55 x 50 x17 mm	

**Very Important Notice**

**- Recharge any time**

There is a rechargeable battery built inside of the hearing amplifier subject, for use under normal condition, it is able to be recharged and discharged 500 times. In order to extend the life of the rechargeable battery, we strongly recommend users to put the hearing amplifier back into charging case for recharging whenever not in use.

Please note that there is no need to recharge your hearing amplifier until it is almost out of power. To charge your hearing amplifier whenever you don't use it will be helpful to maintain the rechargeable battery and extend the usage life for recharging and discharging up to 1000 times maximum.

## Recharge Hearing Amplifiers

### Using AC adapter


1. Please open the AC jack.



2. Please insert the AC Adapter



3. Please observe whether the lamp is green after you insert the socket and turn on the switch.

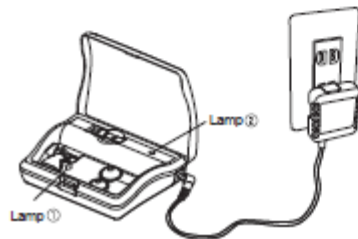


### Using alkaline batteries


1. Please open the lid of battery compartment.



2. Please install two AAA alkaline batteries in the compartment.

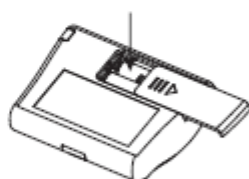

3. Please close the lid of the compartment.

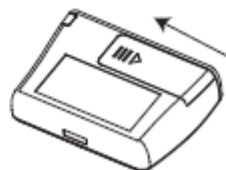

### The Setting method of Charge:

1. Please open the lid of the rechargeable case.




2. The power will be started by twisting the dial of the volume control.

It can't be charged while the power is off.



3. Hold the Hearing Amplifier vertically, and put into the rechargeable case.

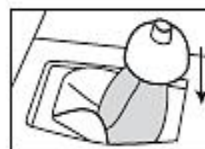


4. Press down lightly and lay down.



5. Please close the lid of rechargeable case.
6. Please observe whether the illuminants of lamps are green after you open the rechargeable case. (If the lamps are not illuminating, it implies that either the location of the Hearing Amplifier is incorrect or the power is off.)



※ The time of charge is approximately 8-10 hours. After the completion of charge, only lamp ② will illuminate. (It indicates incompleteness of charge if two lamps are illuminating at the same time.)

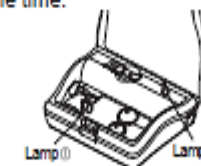

### Take out the Hearing Amplifier.

1. Please open the lid of rechargeable case.



2. Please place the Hearing Amplifier vertically.



3. Please take the Hearing Amplifier out from inside of case..




4. Please cover the lid of rechargeable case.

### Switch ON / OFF

#### The method of the volume adjustment.

1. Please press and twist the volume button volume with your finger.

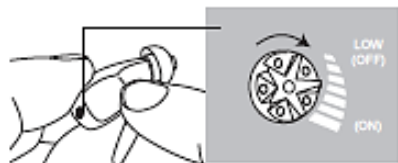

2. The power will be started when you twist to the highest volume, whereas the power will be off after you turn to the lowest volume. (Please turn it off if you have finished using it.)

### Using the hearing amplifier

1. Please turn the power on, and hold the body of Hearing Amplifier.



2. Please use your finger to grasp the microphone and block the hole.




3. Please put it in your ear while you are grasping it.



You will hear the buzzer ringing if you put amplifier into your ear without holding the microphone.

### Maintenance Method

1. Please clean your ear by using cotton swabs before you wear the hearing amplifier. \*\*If the hollow of sound is clogged by your earwax, the quality of volume will be inferior.
2. Please clean out the earwax adhered on the hearing aid. The easiest location on which earwax to be stuffed is around the hollow of sound.

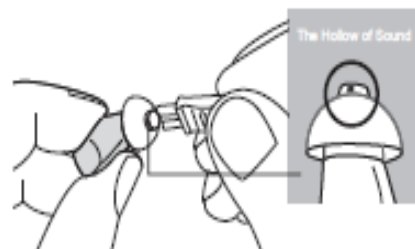


### The Replacement Method of Ear Cap.

- ※ The EAR Cap  
If you can't remove the earwax or there is damage on the body, it's time for a replacement of your ear cap.


1. Please remove the old ear cap from the body as shown.




2. Turn the new ear cap inside out.

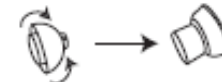

3. Install the new ear cap.

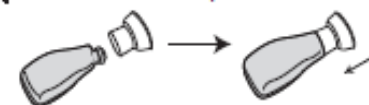

4. Return the ear cap to original state as shown.

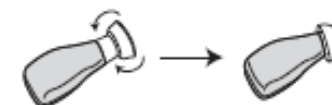

### Storage Method:

Please keep it away from high temperature, humidity, or direct exposure under the sun.

- \* Keep away from children.
- \* Please don't use it while you are sleeping.  
It may cause accidents or physical discomforts.
- \* Please don't wear it during exercise.  
It may cause a breakdown or damage.
- \* Please don't use this product together with your mobile phone or PHS.  
Due to disturbance caused because of the effect of noise resulted from radio wave.
- \* Keep away from water.  
Please don't wear it while taking shower or washing your hair  
Additionally, please be cautious to keep it dry while washing your face or using hair spray, it may cause breakdowns.



## SPECIFICATION & FUNCTION

General title of commodity :	Rechargeable Hearing Amplifier.
Purpose & Function :	For assisting people with hearing difficulties.
Standard Frequency :	2500Hz
Max.output sound level gain	29±5dB 0~29dB
Highest sound pressure level for Input SPL 90dB :	118±5dB below 125dB
Equivalent input noise level:	Below 29dB
Total Harmonic distortion value of reality :	2500Hz: below 3%;800Hz:below 4%;500Hz:below 4%
Battery Current :	Below 0.8mA/2.5V
Battery Type/life expectancy:	Button size Rechargeable Battery. Continuous use: 7~12hours /each recharged time. 500 times in total,(it may be different because of depending on what level of volume you use.)
Suitable Object:	Slight obstacle of hearing.
Operating temperature/ humidity :	+10 <sup>0</sup> C ~ +40 <sup>0</sup> C ,30% ~ 85% RH
Storage temperature / humidity :	-10 <sup>0</sup> C to +60 <sup>0</sup> C ,30% ~ 85% RH
Size:	<Hearing Amplifier> 10.5mm(L)x14.4(W)x24.0mm(H) <Rechargeable Case>96.0(L)x 73.0(W)x 30.0mm(H)
Weight :	<Hearing Amplifier>2.4g <Rechargeable Case>67g
Material :	<Hearing Amplifier> ABS <Rechargeable Case>ABS <Ear Cap> silicone <AC Adapter>ABS

## Troubleshooting

### Can't hear any sound :

#### 1. Does the power switch on?

Please twist the dial of volume adjustment and turn the power on.

#### 2. Did the ear cap clog with earwax?

Please use our accessory --clean brush to clean your ear cap.

#### 3. Have batteries exhausted completely?

Please replace it with new batteries or you can use AC ADAPTER to it.

#### 4. Please fully charge unit before first time use.

UK  
CA



**COOPERS  
OF STORTFORD**

UK Head Office, 11 Bridge Street, Bishop's Stortford, CM23 2JU

Tel: 0844 4824400\*, Fax: 01279 756595

[www.coopersofstortford.co.uk](http://www.coopersofstortford.co.uk)

EU address: D.S.B. LD, 160 Bd. de Fourmies, 59100 Roubaix, France

Tel: (+44) 1279 701269

\*UK Calls cost around 7 pence per minute plus your telephone company's network access.

Please check with your telephone supplier.