

Assembly Instructions

Fig. 1



Fig. 2

1. Open the Coopers Walker by pushing down on the black bar under the seat.
2. Make sure the folding hinges are locked into place to secure the unit correctly. See Fig 1.
3. Place the handlebar into its receptacle and adjust to the desired height. See Fig 2.
4. Place the back rest into its receptacle and ensure it clicks into place.
5. Secure the handlebars with the height adjustment knobs and bolts into position, making sure the knobs are facing outwards. See Fig 2.
6. Hook the basket onto the front of your Walker. See Fig 3.

7. Test the brake tension to ensure safe and proper braking before use.

Using the your Coopers Walker

Please read all caution and warning labels on your Coopers Walker before use. The Coopers Walker is designed as a mobility aid, it allows the user to distribute his/her weight evenly between the handlebars to allow for easier movement over longer distances. The Coopers Walker is designed to fold to allow for easy storage and transport, simply lift up the seat and pull the black bar until the walker has folded taking care not to catch your fingers in the hinges.

- Only use the seat once the parking brake has been applied.
- The brakes have 2 functions **normal braking** and **parking brake**.
- Normal braking is activated by pulling the brake handle up; once the handle is released the brakes are released too.
- The parking brake is activated by pushing the brake handle down; you will hear a click to signal the brake is locked in place. To release the parking brake, pull the handle back upwards to its normal position.

Parts:

- 1x Coopers Walker (A)
- 2x handlebars (B)
- 2x height adjustment knobs (C)
- 2x bolts (D)
- 1x back rest (E)
- 1x basket (F)



Fig. 3

Maintenance

Keep the brake handles clean to allow for smooth operation.

If the brakes feel loose or do not stop properly the cable needs tightening. To adjust the cable, tighten the screw where the cable connects to the handle to the desired tension.

Regularly check all the bolts and screws are properly tightened.

Keep the wheels clean.

Replace the wheels if worn or damaged. When cleaning the Coopers Walker use soapy water and a soft cloth, abrasive cleaners and cloths can scratch the surfaces.

CAUTION:

**Weight limit for Coopers Walker:
110kg (17st 4lb).**

WARNING

Do not use this product for any other purpose than as a walking aid, it is not to be used as a wheelchair of any kind.

Only use the seat once the parking brake has been applied. The Seat is for occasional resting only during walking.

**DO NOT use the rollator as a wheelchair or transport device.
Rollators are not intended to be propelled while seated.**

DO NOT attempt to reach objects if you have to move forward in the seat. Reaching for these objects will cause a change to the weight distribution of the rollator and may cause the rollator to tip, resulting in injury or damage.

**Disposal:**

- Dispose of the packaging at your local recycling centre.
- Dispose of paper and box separately from plastic bags at your local recycling centre.
- Dispose of the item at the end of its lifespan at your local authorised household waste recycling centre.

