

Coopers Lightweight Wheelchair 7707 Instructions

The Coopers Lightweight Transit Chair
should not be operated without assistance



Transit chair closed

Please turn overleaf for instructions

Please read carefully before use and retain for future reference.

Coopers
OF STORTFORD

User and maintenance guide

To open the transit chair push down on the seat rails.

To attach the footrests, align the holes on the footrest with the pegs on the frame and lock into place.

To adjust the foot rests the user should be sitting in the chair with both feet on the foot plates. Loosen the bolt at the bottom of the foot rest, make the necessary adjustments and tighten.

The foot rest should not be lowered to a point where it causes pressure between the thigh and the front of the seat. To ensure sufficient ground clearance the foot rest should be a minimum of 2½in above the ground.

To remove the footrests, release the lock, swing outward and rearward. Lift up and off to remove the footrest.

To fold the transit chair lift the footrests vertically or remove them, hold the front and rear middle of the seat and pull upwards you.

Proper care and maintenance is important to keep your chair in good condition.

At least once a week wipe down the paintwork with a soft cloth. If the chair is exposed to moisture dry immediately. The metal parts should be polished once a month with an auto wax.

Inspect the wheels at least once a month, correct any wobble by loosening the lock nut on the axle, then tighten just enough to remove the wobble.

The tyres will need minimal maintenance but will need to be cleaned periodically. If the tyres become cracked or worn replace immediately.



WARNING

Make sure the chair is in good working condition at all times.

Do not exceed the transit chair's weight capacity 110kg

Make sure the brakes are clean and adjusted correctly before each use.

Always engage the wheel locks when entering or leaving the chair.

Do not move forward in the seat whilst leaning forward. If absolutely necessary make sure the front casters are in the forward position.

Leaning out of the chair without proper assistance may cause tipping.

The wheel chair must be on a level surface when transferring the user to or from the seat.

The foot rests must be no lower than 2½ins from the ground at all times.

Do not stand on or put pressure on the foot rest when entering or exiting the chair.



Car boot storage

Head Office, 11 Bridge Street,
Bishop's Stortford CM23 2JU
Tel: 0844 482 4400
Fax: 01279 756595
www.coopersofstortford.co.uk

Coopers
OF STORTFORD