



## HOW TO FIT:

- Unscrew the side knobs until they extend out from the Raised Toilet Seat.
- Position the Raised Toilet seat onto your toilet. There should be no need to remove your existing toilet seat and lid, however this may depend on the size and type of toilet.
- Tighten the side knobs until the fixing is secure. Do not over tighten. The Raised Toilet Seat should not be able to move once fitted.
- Do not exceed the maximum user weight of 190kg (30 stone approx.)

## HOW TO CARE FOR YOUR ITEM:



- The seat can be removed for thorough cleaning by unscrewing the side knobs.
- Wipe clean as required with a bathroom disinfectant and a soft cloth.
- Do not use any abrasive cleaners or scourers that will scratch.
- Periodically check fixing to ensure it is secure.

## DISPOSAL:

- Coopers of Stortford use recyclable or recycled packaging where possible.
- Please dispose of all packaging, paper, cartons, packaging in accordance with your local recycling regulations.



- Plastics, polybags – this is made from the following recyclable plastic.

Code & Symbol		
Type of Plastic	High-Density Polyethylene (lid)	Polypropylene
Commonly used for	Milk bottles, cleaning product bottles, yoghurt pots, butter tubs and other food containers	Microwaveable food containers, yoghurt pots, butter tubs, disposable plates and cups
Notes	Commonly recycled and collected from household recycling bins. Check with your local authority	Commonly recycled and collected from household recycling bins (packaging tape itself not recyclable but does not interfere with cardboard)