**Parts:**

Rotating seat

Upper Leg

Height adjustment leg

Seat base with seat handle

White rubber feet

Assembly

1. Remove contents from carton.
2. Assemble legs by inserting the height adjustment legs into the white rubber feet. Ensure the feet are secured and positioned as shown in the diagram. Repeat for 3 other legs.
3. Insert the height adjustment leg into upper leg by depressing the brass coloured push pin. Repeat for 3 other legs.

Lay the seat upside-down on a flat surface. Insert each upper leg into the seat depressing the brass coloured push pins and insert into the bottom of the seat. A click will be heard when the leg is securely in place. Repeat for 3 other legs.


Adjusting Leg Height

1. Before use, adjust the height of each leg to your specific needs, making sure all legs are at the same height. When the desired height is reached, lock the legs into place by aligning the brass coloured push pins into the holes.
2. To ensure the legs are locked into place, push down on (or pull up on) the legs.

Rotating Seat

1. To rotate the seat, pull up gently on the handle until the seat is free of one of the 4 locked positions and gently rotate to desired position. The seat can rotate in both directions and has four locking positions which will secure the seat in place.
2. Ensure the seat is in one of the locked positions before sitting on it.

Seat has 4 locking positions and will rotate in both directions

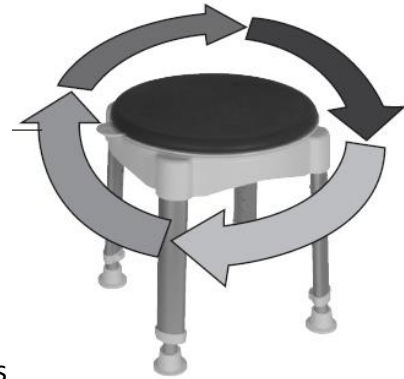

Maintenance

- Wipe dry after every use.
- Clean with a mild cleaning solution and a soft cloth, do not use abrasive cleaners or bleach.
- Before each use check all fixings are securely in place.

WARNINGS / CAUTIONS:

- All four legs with rubber feet **MUST** touch the floor simultaneously at all times. Please check carefully, especially in a bath where the floor surface is not always level.
- Maximum weight limit is 150kg (23st 8lb).
- Do **NOT** stand on the stool.
- Keep away from children, this is not a toy.
- Not for use in saunas or extreme temperature conditions.
- Not for commercial use.

DISPOSAL

- Coopers of Stortford use recyclable or recycled packaging where possible.
- Please dispose of all packaging, paper, cartons, packaging in accordance with your local recycling regulations.







Rotating Bathroom Seat 8804 Instructions for Use

Please read and retain these instructions for future reference

v004 11/06/20

- Plastics, polybags – this is made from the following recyclable plastic.

Code & Symbol	 PVC	 LDPE	 PP	 OTHER
Type of Plastic	Polyvinyl Chloride	Low Density Polyethylene	Polypropylene	PU
Commonly used for	Pipes, construction material, flooring, windows, guttering, blister packs	Shopping bags, bread bags, squeeze bottles, stretch/pallet wrap, bubble wrap, multi-pack can holders	Microwaveable food containers, yoghurt pots, butter tubs, disposable plates and cups	CD's, crisp packets, various flexible packaging, baby bottles, sunglass lenses
Notes	Not usually collected from household recycling bins, Sometimes recycled at local waste management centres.	Usually recyclable supermarket deposit points. Check other plastic films as any non-polythene film e.g. cling film, crisp packets film lids	Commonly recycled and collected from household recycling bins (packaging tape itself not recyclable but does not interfere with cardboard)	Not often or widely recycled

- At the end of the product's lifespan please check with your local council authorised household waste recycling centre for disposal.

Head Office, 11 Bridge Street,
Bishop's Stortford CM23 2JU
Tel: 0844 482 4400*
Fax: 01279 756 595
www.coopersofstortford.co.uk

Coopers
OF STORTFORD

*Calls cost 7 pence per minute plus your
telephone company's network access charge