

Read carefully before use and retain for future reference.

**MALE USE:** Without attachment

**FEMALE USE:** With attachment

Note: The attachment can be used either way round, with the cup at either the front or rear, as chosen by the user.

## **WARNING**

- Take care during use and during transportation to avoid spills.
- Maximum capacity: Approx. 1 litre
- Do not exceed capacity.
- Always carry by the carry handle.
- Ensure you do not contravene public order legislation when using this item.
- Ensure the cap is securely fitted after use.
- Empty into a toilet at the earliest convenience after use.

## **CLEANING**

- Clean after each use using bleach or household toilet cleaner.
- Always empty fully before cleaning.

## **PACKAGING**

- Dispose of packaging at your local recycling centre.
- Dispose of paper and carton separately from plastic bags at your local recycling centre.
- Dispose of item at the end of its lifespan at your local authorised household waste recycling centre.