

Folding Lightweight Walker 9836

Instructions

Opening the Walker

1. As shown in figure 1 below, pull the handles as far apart from each other as possible.
2. As shown in figure 2 below, push the locking pad down to lock the walker into the open position.



Figure 1



Figure 2

Closing the Walker

1. As shown in figure 3, you will see the locking pad lever.
2. As shown in figure 4, pull up on the locking pad lever to release the locking mechanism, then fold the walker shut.



Figure 3

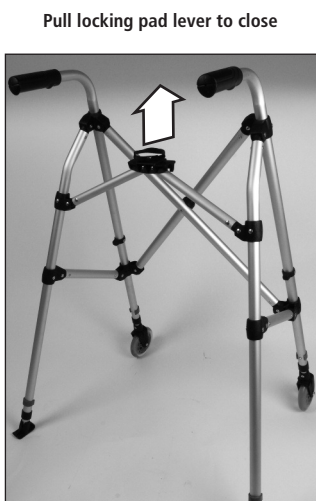


Figure 4

Height Adjustment

1. As shown in figure 5, push the silver pin in, which is located behind the clip on each leg, then remove this clip.
2. You can then slide the leg up or down until you have reached the desired height.
3. Re-fix the clip by aligning the silver pin with the hole in the leg and clip into place.
4. Repeat steps 1 to 3 above on each leg. Make sure each leg is at the same desired height as each other.

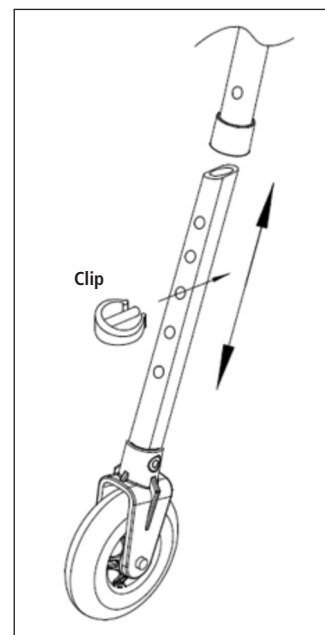



Figure 5

Worn Feet Replacement

PLEASE NOTE this information is for future reference for replacing feet that have worn over time, you do not need to replace the feet on a newly purchased walker.

1. As shown in figure 6, place a regular screwdriver in the recess slots located at the top of the feet front and back tabs.
2. Bend both the front and back tabs out and away from the leg.
3. Pull the skid in a downward position to remove the worn out foot.
4. Using the new foot, bend the front and back tabs outward and slide the foot upward onto the tube until the pin located on the front tab engages the hole shown in the picture below.

Figure 6

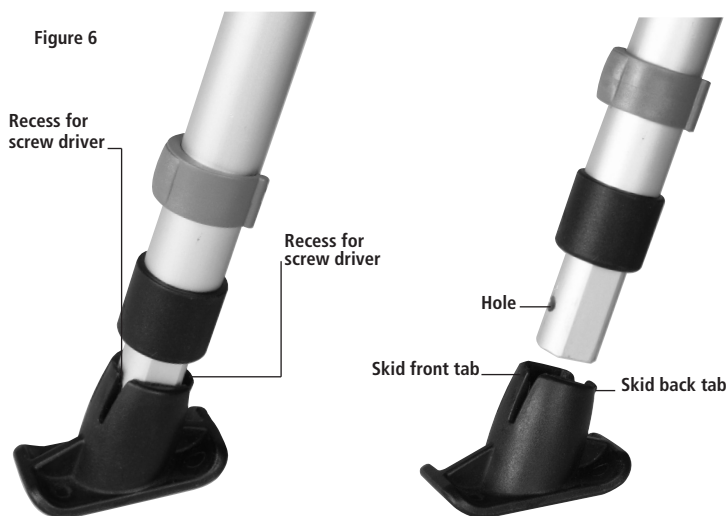


Please read carefully before use and retain for future reference.

Coopers
OF STORTFORD

Using your Folding Lightweight Walker

Please read all caution and warnings for your walker before use. The walker is designed as a mobility aid, it allows the user to distribute his/her weight evenly between the handlebars, to allow for easier movement over longer distances.

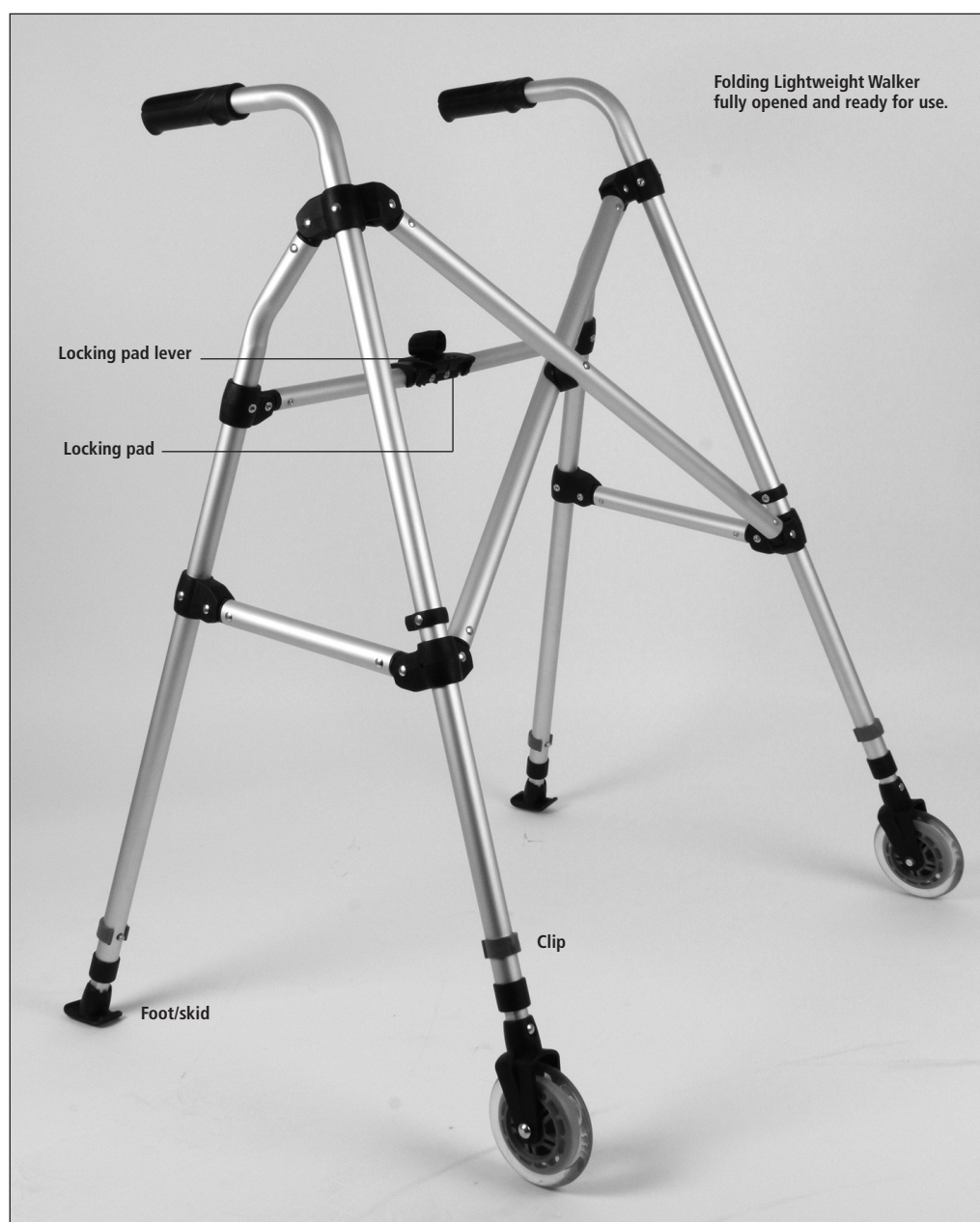

DISPOSAL

Dispose of all paper, cartons and plastic in accordance with your local recycling regulations. At the end of the product's lifespan please dispose of it at an authorised household waste recycling centre.

WARNING: Please read the following before using this product:

1. Regularly check the walker to make sure there are no broken or loose parts.
2. Individuals should not use this product if they are in a high risk condition. High risk users include those who are not mentally coherent (including those individuals under medication).
3. This walker has a maximum weight limit of 100kg, it is not suitable or safe for users above this weight limit to use.
4. If you are suffering from an ailment or injury or are taking medications that affect your balance or ability to stand or sit down, you should first consult with a physician before using this product. If the product is not used properly, serious injury or harm could result.
5. Children should not be allowed to hang from or play with this product.
6. Always ensure that you use this walker on a firm and level surface.



Head Office, 11 Bridge Street,
Bishop's Stortford CM23 2JU
Tel: 0844 482 4400
Fax: 01279 756595
www.coopersofstortford.co.uk

Coopers
OF STORTFORD