



This is not a medical device but an assistive product to help get up from sitting position

LIST OF PARTS AND CONTENTS:

- A Handle
- B Seat
- C Round Screw
- D Plastic Screw Cover
- E Metal fixing bracket with Plastic screw
- F Hex wrench

A 2x	
B 1x	
C 2x	
D 3x	
E 3x	
F 1x	

USER INSTRUCTIONS (DETAILS OF HOW TO USE THE PRODUCT):

Your new Raised Toilet Seat offers optimum functionality and comfort.

Read these instructions carefully before assembling and using the product. Retain the instructions for future reference. If you pass the product on, please also pass on these instructions. The manufacturer and importer assume no liability for incorrect use of the product or non-observance of the information in these instructions.

ASSEMBLY INSTRUCTIONS:



Fig.A



Fig.B

**COOPERS
OF STORTFORD**

UK Head Office, 11 Bridge Street, Bishop's Stortford, CM23 2JU

Tel: 0844 4824400*, Fax: 01279 756595

www.coopersofstortford.co.uk

EU address: D.S.B. LD, 160 Bd. de Fourmies, 59100 Roubaix, France

Tel: (+44) 1279 701269

*UK Calls cost around 7 pence per minute plus your telephone company's network access.

Please check with your telephone supplier.

1. Remove or lift up your current toilet seat and lid.
2. Fix the handle (A) to the seat(B).
3. Slide one of the plastic screw covers (D) down over one of the three pre-inserted plastic brackets on the bottom of the seat, making sure that the bracket sits flat.
4. Repeat step 3 on the remaining 2 pre-inserted plastic brackets on the bottom of the seat.
5. Insert the metal fixing bracket with plastic screw (E) into the holes in the back of the pre-inserted plastic brackets on the bottom of the seat. Ensuring that the plastic screw lines up with the round hole.
6. Repeat step 5 on the remaining pre-inserted plastic brackets on the back of the seat.
7. Sit the item on top of your toilet basin and position so it sits straight. Tighten the 3 plastic screws (E) on the base of the seat, until the item sits securely and firmly on top of your toilet basin.

WARNINGS / CAUTIONS: 

- Do not use any caustic or abrasive cleaning agents or sponges. These can damage the surface.
- Keep the packing material away from children (risk of suffocation).
- Keep children away from the toilet aid. They could attempt to stand on it or pull themselves up and fall over.
- Ensure that another person is close by. This person can help you with sitting down or standing up if necessary.
- The toilet seat supports maximum weight of 100kg.
- The handle is designed to assist your natural motion, it can't support your full body weight! Inspect the toilet seat before use and ensure that it is fastened securely.

DISPOSAL

- Coopers of Stortford use recyclable or recycled packaging where possible.
- Please dispose of all packaging, paper, cartons, packaging in accordance with your local recycling regulations.
- At the end of the product's lifespan please check with your local council authorised household waste recycling centre for disposal.
- Plastics, polybags – Contains the following recyclable plastic.



Code & Symbol	 HDPE	 PP	 OTHER
Type of Plastic	High-Density Polyethylene	Polypropylene	EVA
Commonly used for	Milk bottles, cleaning product bottles, yoghurt pots, butter tubs and other food containers	Microwaveable food containers, yoghurt pots, butter tubs, disposable plates and cups	CD's, crisp packets, various flexible packaging, baby bottles, sunglass lenses
Notes	Commonly recycled and collected from household recycling bins. Check with your local authority	Commonly recycled and collected from household recycling bins (packaging tape itself not recyclable but does not interfere with cardboard)	Not often or widely recycled