

# Shower Seat with Arms 10470

# Instructions for Use

Please read and retain these instructions for future reference

v004: 29/05/19



## Technical Details:

Max loading:150 kg

### 1. Basic Parameters and size



### 2. Parts

Unit name	Quantity	Picture
① backrest	1PC	

\*Calls cost 7 pence per minute plus your telephone company's network access charge

Head Office, 11 Bridge Street,  
Bishop's Stortford CM23 2JU  
Tel: 0844 482 4400\*  
Fax: 01279 756 595  
[www.coopersofstortford.co.uk](http://www.coopersofstortford.co.uk)


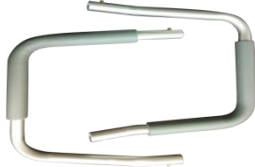






**Coopers**  
OF STORTFORD

# Shower Seat with Arms 10470

# Instructions for Use

Please read and retain these instructions for future reference

v004: 29/05/19

<p>② backrest tube</p>	<p>2PCS</p>	
<p>③ armrest tube</p>	<p>2PCS</p>	
<p>④ seat carriage</p>	<p>2pcs</p>	
<p>⑤ seat board</p>	<p>1pc</p>	
<p>⑥ U-shaped tube</p>	<p>1pc</p>	
<p>⑦ M6*57 screw+M6 washer Use to assemble seat carriage (4) and U-shaped tube (6))</p>	<p>3pcs</p>	
<p>⑧ M6*30 screw Use for screwing backrest tubes (2) into backrest (1).</p>	<p>4pcs</p>	
<p>⑨ M6*35 stainless screw+M6 washer Use to hold front of seat in place.</p>	<p>2pcs</p>	

Please ensure you use the correct screws for the correct parts.

Do not fully tighten screws initially. Once all parts are in place, go round each screw and tighten up.

\*Calls cost 7 pence per minute plus your telephone company's network access charge


Head Office, 11 Bridge Street,  
Bishop's Stortford CM23 2JU  
Tel: 0844 482 4400\*  
Fax: 01279 756 595  
www.coopersofstortford.co.uk

**Coopers**  
OF STORTFORD


## 2. Assembly Introductions

**Step 1.** Assemble the seat carriage , put the seat board on the frame



- a) Assemble the 2 seat carriages (4).
- b) Secure the U-shaped tube (6).
- c) Add the seat board.

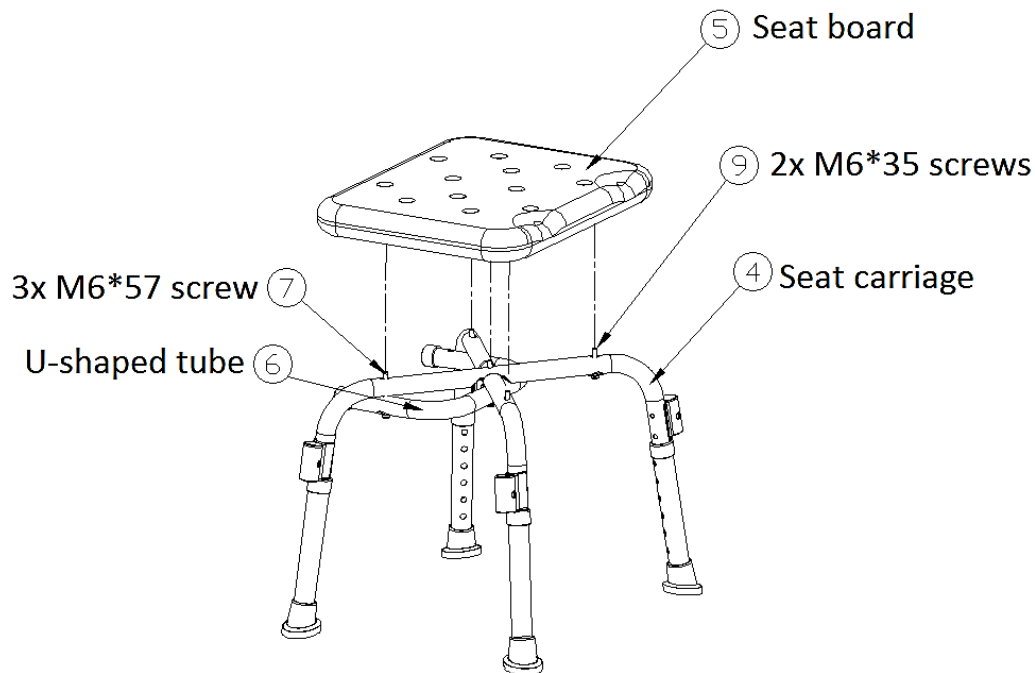

**Step 2:** Assemble the backrest board onto the 2 Backrest tubes (2) and secure with the 4x M6&30 screws (8).

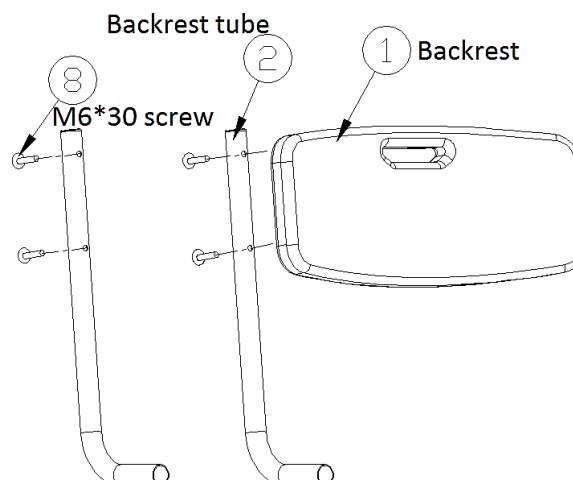


**Step 3:** Insert the backrest to the U shape tube until it clicks into position.

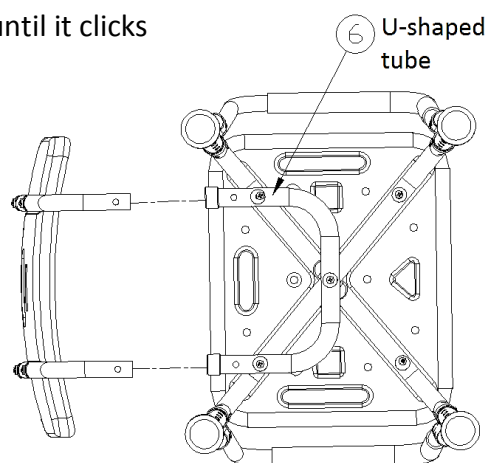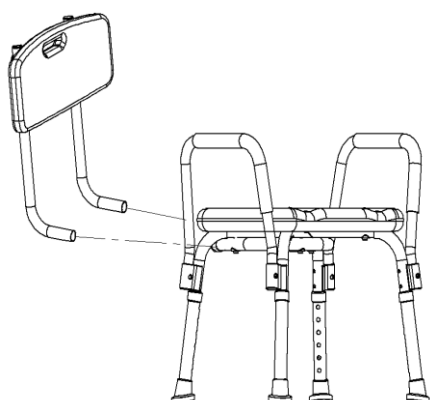

**Step 4:** Insert the armrests to the seat carriage until it clicks into position

Tighten all screws.

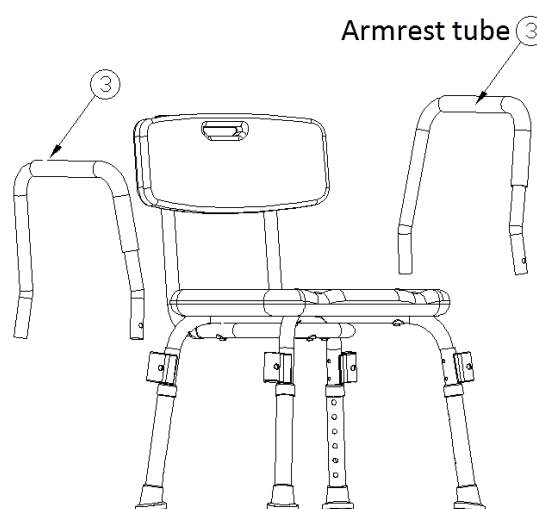

## INSTRUCTIONS:

Before getting into the shower, ensure the Shower Seat is securely positioned and will not slip.

## CLEANING INSTRUCTIONS:

- Wipe surfaces with a damp cloth.
- Allow to dry fully before next use.

## SAFETY INFORMATION:

- Keep away from children. Children could slip or injure themselves playing on the Shower Seat.
- Where possible ensure that another person is nearby to help assist with standing and sitting if required.
- Check before use that the seat is positioned securely in the shower.
- Use non-slip mats where possible to reduce the risks of slips and falls.
- Use only for intended use.

## DISPOSAL:

- Dispose of all packaging, paper, cartons and plastic, plastic bags in accordance with your local recycling regulations.