

Large Display Body Scales 10573 Instructions for Use

Please read and retain these instructions for future reference

v002: 03/01/2017



DIAL ZERO CONTROL WHEEL

USER INSTRUCTIONS:

- Please ensure that the needle is exactly pointing at zero on the dial to avoid a false reading.
- To adjust the needle, use the adjusting wheel on the front of the scale.
- Maximum Weight: 135KG (21ST)
- Use only on a level floor surface.
- For most accurate results, use at a similar time of day and with all garments and footwear removed.
- Stand gently; display will vary if you move.
- Adjust the Dial to zero by rotating the dial zero control wheel clockwise or anticlockwise as necessary before each use.

WARNINGS / CAUTIONS:

- Do NOT use abrasives when cleaning.
- Always stand with feet either side of the centre of the scale.
- Not recommended for being used on carpets with a deep pile.

CLEANING INSTRUCTIONS:

- Wipe clean with a damp cloth.

DISPOSAL:

- Dispose of all packaging, paper, cartons and plastic, plastic bags in accordance with your local recycling regulations.
- At the end of the product's lifespan please dispose of it at an authorised household waste recycling centre.

Head Office, 11 Bridge Street,
Bishop's Stortford CM23 2JU
Tel: 0844 482 4400*
Fax: 01279 756595
www.coopersofstortford.co.uk

Coopers
OF STORTFORD

*Calls cost 7 pence per minute plus your telephone company's network access charge.