
## Cane Safe 10689

# Instructions for Use

Please read and retain these instructions for future reference

v001: 19/12/16

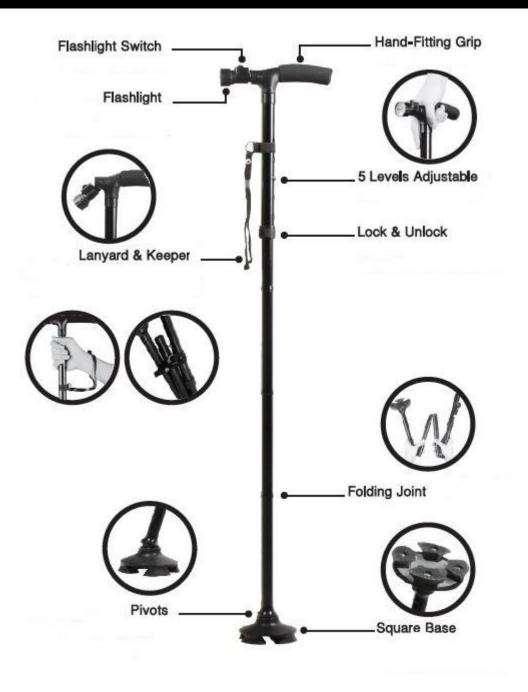

### Before using

- Remove all plastic films.
- Unscrew front part of flashlight then insert 4 new batteries. (LR44 type included)

### **User Instructions**

- To fit and fold the cane, pull 4 parts apart and push fit them. Fold and hold parts together to be held together by lanyard.
- To assemble the cane, plug 4 parts together for safety, all bolts should be locked in proper position.

Head Office, 11 Bridge Street, Bishop's Stortford CM23 2JU Tel: 0844 482 4400 Fax: 01279 756595 www.coopersofstortford.co.uk



## Cane Safe 10689

# Instructions for Use

Please read and retain these instructions for future reference

- To adjust the height, pull or push the handle to any desired level.
- To turn on/off flashlight, press the button once to turn on flashlight and press it again to turn off.
- To adjust the flashlight angle, tilt the front part of the lamp up or down as desired.

#### Caution

- Do not look into the flashlight beam. Keep the beam away from people and animals.
- Maximum weight is 112kg.
- Crashing or dropping may cause damage to the cane.
- Do not use when walking on ice.
- Do not repair the cane yourself.

### BATTERIES

- Keep batteries away from children and pets.
- Do not mix used and new batteries.
- Remove the batteries if left for long periods.
- Never dispose of batteries in a fire.
- Battery disposal, spent or expired batteries must be properly disposed of and recycled in compliance with local regulations. For detailed information, contact your local authority.
- Follow the battery manufacturer's safety, usage, and disposal instructions.

### DISPOSAL:

- Dispose of all packaging, paper, cartons and plastic, plastic bags in accordance with your local recycling regulations.
- At the end of the product's lifespan please dispose of it at an authorised household WEEE waste recycling centre.



\*Calls cost 7 pence per minute plus your telephone company's network access charge.

